## Personenwaage Iris

Bathroom Scales Iris | Báscula personal Iris  
Pèse-personne Iris | Bilancia pesapersona Iris  
Waga osobowa Iris

Deutsch

English

Español

Français

Italiano

Polski

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BE 1700

Operating instructions | Instrucciones de operación | Mode d'emploi  
Istruzioni per l'uso | Instrukcja obsługi

# Liebe Kundin, lieber Kunde !

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# Dear Customer!

You have chosen to purchase a high-quality product from the brand ADE, which combines intelligent functions with an exceptional design. These bathroom scales make it possible for you to determine and monitor your body weight. The many years of experience of the brand ADE ensures a high technical standard and proven quality.

We hope you enjoy weighing yourself and wish you success in reaching your goal weight!

Your ADE Team

## General Information

---

### About this manual

This operating manual describes how to safely use and care for your products. Keep this operating manual in a safe place in case you would like to refer to it later. If you pass this item on to someone else, pass on this manual as well.

Failure to observe the recommendations in this operating manual can lead to injuries or damage to your products.

### Explanation of symbols

This symbol, combined with the word DANGER, warns against serious injury.

This symbol, combined with the word WARNING, warns against moderate and mild injuries.

This symbol, combined with the word NOTE, warns against material damage.

This symbol signifies additional information and general advice.

# Table of Contents

---

General Information .....	12
Intended Use.....	14
Safety .....	14
At a glance (scope of delivery) .....	16
Scope of delivery .....	16
Start-up .....	17
Weighing .....	18
Resetting the scales.....	18
Cleaning.....	18
Fault/Remedy.....	19
Technical Data .....	19
Declaration of Conformity .....	20
Guarantee .....	20
Disposal .....	21

## Intended Use

---

The bathroom scales are designed for the weighing of people in the home. It is not suitable for use in clinics, medical practices, etc.

The maximum carrying capacity of 180 kg must not be exceeded.

## Safety

---

- These scales can be used by children 8 years of age and above, as well as by persons with reduced physical, sensory, or mental capacities or lacking experience and knowledge, if they are supervised or have been instructed in the safe use of the scales and understand the dangers that may arise.
- Children must not play with the scales.
- Cleaning and user maintenance must not be carried out by children, unless they are 8 years of age or older and are being supervised.
- Batteries, when swallowed, can be life-threatening. You should therefore keep the scales and battery out of reach of small children. If a battery is swallowed, medical assistance must be sought immediately.

### **DANGER for children**

- Keep children away from packaging material. Risk of choking if swallowed. Also beware that children do not pull the packaging material over their heads.

### **RISK of fire/burning and/or explosion**

- There is a risk of explosion if batteries are replaced incorrectly. Replace the batteries using only the same or an equivalent type of battery. Pay attention to the "Technical Data".
- Batteries must not be recharged, reactivated by other means, dismantled, thrown into fire, or short-circuited. Otherwise there is an increased risk of explosion and of leakage; furthermore, gases may escape.
- Avoid contact with skin, eyes, and mucosa. If you come into contact with battery acid, flush the affected site immediately with plenty of clean water and see a doctor promptly.

## **WARNING against risk of injury**

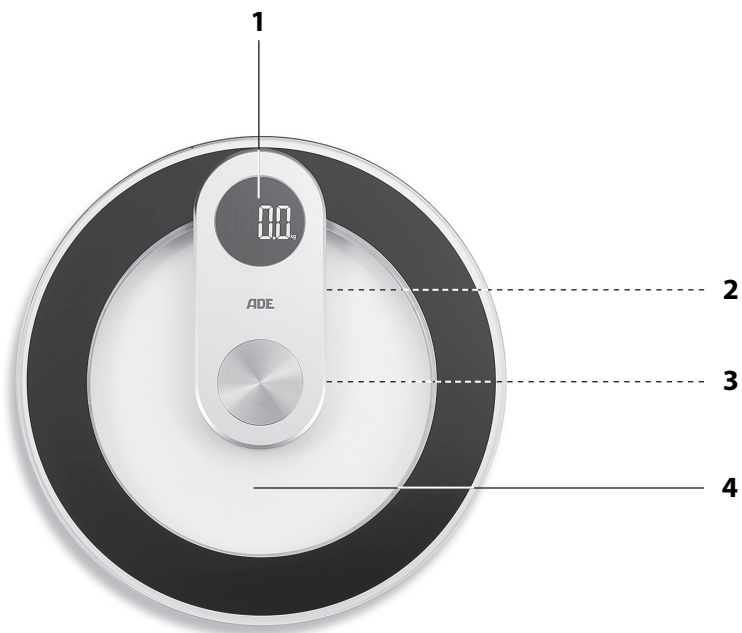
- Do not step onto the outer edge of any one side of the scales. Risk of tipping!
- Do not step onto the scales with wet feet or when its surface is damp. Risk of slipping!
- If the scales fall and shatter, there is a risk of injury from glass splinters.

## **BEWARE of material damage**

- Do not submerge the scales in water or other liquids.
- Place the scales on a stable, even surface.
- Do not overload the scales. Pay attention to the “Technical Data”. Place objects on the scales exclusively for the purpose of weighing.
- Protect the scales from sharp jolts and vibrations.
- Remove the batteries from the scales if they have been used up or if you are not going to use the scales for a long time. You will thus avoid damage that may result from the leakage of battery acid.
- When inserting and changing batteries, pay attention to the correct polarity (+/-).
- Do not subject the batteries to extreme conditions by, for example, storing them on heaters or under direct sunlight. Increased risk of leakage!
- Clean the battery and device contacts, as needed, before inserting the battery.
- Only insert batteries of the same type; do not use different types or used and new batteries with each other.
- Do not make any modifications to the scales. Only allow repairs to be carried out at a specialist workshop. Repairs carried out incorrectly can considerably endanger the user.
- Do not place the scales in the vicinity of devices that emit electromagnetic radiation, such as mobile phones, wireless devices, and the like. This can lead to incorrect display indicators or malfunctions.
- Do not place any objects on the scales when you are – for example – storing them in a cupboard. The sensitive measuring electronics in the scales could be damaged by heavy objects.
- There are feet on the underside of the scales. Since surfaces are made from various materials and are treated with diverse care products, the possibility cannot be completely excluded that some of these substances contain ingredients that attack and soften the feet of the device. If necessary, place an anti-slip underlay beneath the scales.

## At a glance (scope of delivery)


---

- 1 Display
- 2 Control panel to change the unit of measure (underside)
- 3 Battery compartment (underside)
- 4 Weighing surface

## Scope of delivery

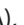---

- Bathroom scales
- Battery type LR03 (AAA), 1,5 V  (4x)
- Operating Manual

# Start-up

---

## Inserting/changing batteries

In order to use the scales, you will need 4 batteries of type LR03 (AAA), 1.5 V (included). The battery compartment is located on the underside of the device.

1. Slide off the battery compartment cover in the direction of the arrow.
2. Insert the batteries as shown on the base of the battery compartment. Pay attention to the correct polarity (+/-).
3. Replace the battery compartment lid and slide it towards the edge of the scale all the way to the stop.

Notes on changing the battery:

- A change of batteries becomes necessary when “LO” appears on the display, or when nothing at all is displayed.
- Only use the type of battery that is specified in the “Technical Data”.
- Always change all batteries at the same time.
- Dispose of batteries in an environmentally appropriate way (see “Disposal”)
- Calibrate the scale, see „Setting the scale“.

## Choosing the unit of measurement

When the scales are delivered, the unit of measurement is set to “kg” as standard. However, the measurement can also be displayed in the English unit of measure „lb” (pounds).

- Press the control panel on the back to set the desired unit of measure. Pay attention to the display indicator.

## Positioning the scales

1. Place the scales on an even, firm, and dry surface. Do not place the scales on carpet.
2. First, briefly step onto the middle of the weighing surface to calibrate the scale. As soon as the display shows „0.0”, calibration is complete. This calibration must be performed after each battery change.

# Weighing

---

## **WARNING** against injuries

- Do not step onto the scales with wet feet or when its surface is damp. Risk of slipping!
- Position yourself on the scales. Remain standing still and do not hold on to anything.

The scale switches on and the display flashes shortly afterwards:

The weight measurement is displayed in the selected unit of measure.

A few seconds after you step off the weighing surface, the scales will switch off automatically.

- The weight is displayed for up to approx. 10 seconds after leaving the scale. When you step on the weighing platform using slight force, the display shows „0.0“ and you can weigh yourself again.
- Weigh yourself with as little clothing on as possible.
- Try to weigh yourself always at the same time and under consistent conditions in order to obtain comparable values.

# Resetting the scales

---

Reset the scales to factory settings if unusual or obviously incorrect values are shown on the display.

1. Remove the batteries from the battery compartment and wait a moment.
2. Re-insert the batteries. Pay attention to the correct polarity of the batteries (+/-).

After switching the scales on, you can weigh yourself as usual.

# Cleaning

---

## **NOTE** on material damage

- Do not immerse the scales in water or other liquids.
- Do not under any circumstances use harsh, scraping, or abrasive cleaning agents or the like to clean the scales. If you do, the surface could be scratched.
- Wipe down the scales as needed using a lightly dampened, soft cloth.
- Disinfect the weighing surface from time to time using a suitable disinfecting agent (available at specialist retailers and pharmacies).

## Fault/Remedy

Fault	Possible Cause
No function	<ul style="list-style-type: none"> <li>Are the batteries empty or have they been inserted incorrectly?</li> </ul>
The scales appear to be displaying false weighing results	<ul style="list-style-type: none"> <li>Are the scales standing on a firm, even surface?</li> <li>Is a device that can cause electromagnetic disturbances located in the vicinity of the scales?</li> <li>Remove the batteries from the battery compartment and wait a moment. Re-insert the batteries.</li> </ul>
"Err" is shown on the display	<ul style="list-style-type: none"> <li>The maximum capacity of the scales (180 kg) has been exceeded.</li> </ul>
"LO" is shown on the display.	<ul style="list-style-type: none"> <li>The batteries are empty (see "Inserting/ changing batteries").</li> </ul>
"C" is shown on the display.	<ul style="list-style-type: none"> <li>An error occurred while you were on the scale. Repeat the process.</li> </ul>

## Technical Data

Model:	Bathroom Scales BE 1700 „Iris“
Battery:	4x LR03 (AAA) /1,5 V 
Rated current:	< 30 mA
Measuring range:	max. 180 kg
Variance:	1 % +/- 100 g
Indicator division:	100 g
Weight:	approx. 1.7 kg
Dimensions of scales:	ca. Ø 330 x 27 mm
Distributor (no service address):	Waagen-Schmitt GmbH Hammer Steindamm 27-29 22089 Hamburg, Germany

Our products are constantly being developed and improved. For this reason, design and technical modifications are possible at any time.

# Declaration of Conformity

---

These scales have been manufactured in accordance with the harmonised European standards.

It conforms to the provisions of the EC directives cited below:

- EMC Directive 2014/30/EU
  - RoHS Directive 2011/65/EU.
- in the currently valid versions.

This declaration loses its validity if modifications are made to the products without our approval.

Hamburg, March 2017  
Waagen-Schmitt GmbH  
Hammer Steindamm 27-29  
22089 Hamburg, Germany

## Guarantee

---

The company Waagen-Schmitt GmbH guarantees for 3 years from the purchase date the cost-free rectification of defects on the basis of material or fabrication errors through repair or exchange.

In the event of a guarantee claim, please return the product with the proof of purchase (stipulating the reason for claim) to your dealer.

# Disposal

---

## Disposing of the packaging

Dispose of the packaging with similar materials. Put cardboard with waste paper, films with recyclable materials.

## Disposing of the product

Dispose of the product in accordance with the regulations that apply in your country.

### **Devices must not be disposed of in normal household waste.**

At the end of their life, the products must be disposed of in an appropriate way. In this way, valuable materials contained in the device will be recycled and the burden on the environment avoided. Hand in the old device at a collection point for electronic waste or at a recycling depot.

For more information, contact your local waste disposal firm or your local administrative body.

### **Recyclable and disposable batteries do not belong in household waste.**

All batteries must be handed in at a collection point in your community, suburb, or at the retailer. Such batteries can thus be disposed of in an environmentally friendly way.

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