Körperanalysewaage BA 1600 | 1800 FITvigo

Body Analyser Scale | Báscula de análisis corporal

Balance d'analyse corporelle | Bilancia con analisi corporea

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FITvigo

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Operating instructions | Instrucciones de operación | Mode d'emploi
Istruzioni per l'uso | Instrukcja obsługi

Liebe Kundin, lieber Kunde !

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Dear Customer!

You have chosen to purchase a product from the FITvigo program from ADE: Activity trackers, bathroom and body analyser scales, as well as blood pressure monitors are equipped with a Bluetooth function in order to transfer all data to the FITvigo app. FITvigo is free and, alongside clear progress monitoring, offers additional motivation and stimulation for more well-being. At the same time, the many years of experience of the brand ADE guarantee the high quality of the product and the app.

We hope you enjoy your FITvigo product.

Your ADE Team

General Information

About this manual

This operating manual describes how to safely use and care for your products. Keep this operating manual in a safe place in case you would like to refer to it later. If you pass this item on to someone else, pass on this manual as well.

Failure to observe the recommendations in this operating manual can lead to injuries or damage to your product.

Explanation of symbols

This symbol, combined with the word DANGER, warns against serious injury.

This symbol, combined with the word WARNING, warns against moderate and mild injuries.

This symbol, combined with the word NOTE, warns against material damage.

This symbol signifies additional information and general advice.

Table of Contents

General Information	18
Intended Use.....	20
Safety	20
Overview	22
Scope of delivery	22
Start-up	23
This is how your body analyser scale works	24
Measuring your weight.....	25
Entering personal data.....	25
Automatic user recognition.....	26
Body analysis.....	26
Using the FITvigo app.....	29
Cleaning.....	29
Fault/Remedy.....	30
Technical Data	31
Declaration of Conformity	32
Guarantee	32
Disposal	32

Intended Use

The body analyser scale is intended for the weighing of individuals in a private setting. It is not suitable for use in clinics, medical practices, etc.

Likewise, the body analysis function is designed only for private use. The scale is not suitable for use in the professional sporting sphere.

The maximum carrying capacity of 180 kg must not be exceeded.

Do not use the scale if you ...

- ... are pregnant,
- ... have a cardiac pacemaker,
- ... are a dialysis patient,
- ... suffer from oedemas,
- ... have implants, e.g. artificial limbs, metal plates, etc.

If in doubt, consult your GP.

Safety

- This item can be used by children 8 years of age and above and by persons with reduced physical, sensory, or mental abilities or lacking experience and knowledge, if they are supervised or have been instructed in the safe use of the item and understand the dangers that may arise.
- Children must not play with the item.
- Cleaning and user maintenance must not be carried out by children, unless they are 8 years of age or older and are being supervised.
- Batteries, when swallowed, can be life-threatening. You should therefore keep the batteries and scale out of reach of small children. If a battery is swallowed, medical assistance must be sought immediately.

DANGER for children

- Keep children away from packaging material. Risk of choking if swallowed. Also make sure that children do not pull the packaging material over their heads.

RISK of fire/burning and/or explosion

- Risk of explosion if batteries are improperly exchanged. Replace the batteries only with one of the same or an equivalent type. Pay attention to the “Technical Data”.
- Batteries must not be recharged, reactivated by other means, dismantled, thrown into fire, or short-circuited.
- Avoid contact with skin, eyes, and mucosa. If contact is made with the battery fluid, rinse the affected areas immediately with plenty of clean water and see a doctor straight away.

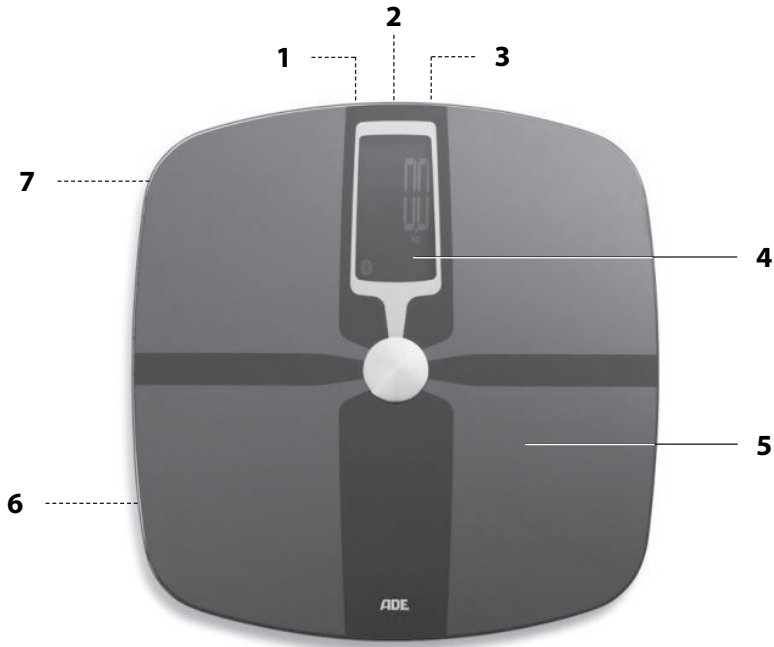
WARNING against risk of injury

- Do not step onto the outer edge of any one side of the scale. Risk of tipping!
- Do not step onto the scale with wet feet or when its surface is damp. Risk of slipping!
- The weighing surface is made of glass. If you drop the scale, there is a risk of the glass breaking and thus a risk of injury from glass splinters.

BEWARE of material damage

- Place the scale on a stable, even surface.
- Do not overload the scale. Pay attention to the “Technical Data”. Place yourself on the scale only for weighing purposes.
- Protect the scale from sharp jolts and vibrations.
- Remove the batteries from the scale if they have been used up or if you are not going to use the scale for a long time. You will thus avoid damage that may result from the leakage of battery acid.
- Pay attention to the correct polarity (+/-) when exchanging the battery.
- Do not expose the batteries to any extreme conditions, e.g. by storing them on heaters or under direct sunlight. Increased risk of leakage!
- Clean the battery and device contacts, as needed, before inserting the battery.
- Do not make any modifications to the item. Only allow repairs to be carried out at a specialist workshop. Repairs carried out incorrectly can considerably endanger the user.
- There are feet on the underside of the scale. Since floor surfaces are made of various materials and are treated with diverse care products, it cannot be completely ruled out that some of these substances contain constituents that attack and soften the feet of the device. If necessary, place an anti-slip underlay beneath the scale.

Overview

- 1 Button ◀: Select memory cell, reduce data
- 2 **SET** button: Call up setting mode, store entries, call up stored measurement data
- 3 Button ▶: Select memory cell, increase data
- 4 Display
- 5 Weighing surface with ITO coating
- 6 Battery compartment
- 7 Unit and Bluetooth button: Choose unit of measurement, connect scales with the app

Scope of delivery

- Scale
- Battery type R03 (AAA), 1.5 V --- (4x)
- Operating instructions

Start-up

Inserting/changing batteries

For use 4 batteries of type R03 (AAA), 1.5 V are required (included). The battery compartment is located on the underside of the device.

1. Press the closing flap of the battery compartment cover in the direction of the cover.
2. Remove the battery compartment cover.
3. Insert the batteries as depicted on the bottom of the battery compartment. Pay attention to the correct polarity (+/-).
"8888" appears briefly on the display, then "0.0 kg".
4. Put the battery compartment cover back in place.

Notes on changing the battery:

- A change of batteries becomes necessary when "" appears on the display, or when nothing at all is displayed.
- Only use the type of battery that is specified in the "Technical Data".
- Always exchange all of the batteries at the same time.
- Dispose of batteries in an environmentally appropriate way (see "Disposal").

Choosing the unit of measurement

When the scale is delivered, the unit of measurement is set to "kg" as standard. However, the measurement result can also be displayed in the English units of measurement, "lb" (pound) or "st" (stone).

- Press the Unit and Bluetooth button on the rear side of the scale repeatedly. Pay attention to the display indication.

Install app

1. Make sure your smartphone is connected to the internet.
2. Download the "FITvigo" app in the Apple App Store (iOS) or Google Play Store (Android).

- Depending on the settings of your smartphone, additional costs may be incurred for the internet connection or data transfer.
- We recommend downloading over a wireless network.
- The FITvigo app itself is free.

Connect the device with the app (pairing)

1. Activate Bluetooth on your smartphone.
2. Start the app and follow the instructions. Pay attention to the information and explanations displayed.
The scale will be connected to the FITvigo app after a short while.

Positioning the scale

- Place the scale on an even, firm, and dry surface. Do not place the scale on carpet.

This is how your body analyser scale works

Your new body analyser scale works using so-called BIA-technology (Bioelectric Impedance Analysis). This involves an undetectable, completely harmless and weak current flowing through your body.

The measurement takes place by means of a special coating which has been applied to the weighing surface.

- The current flows relatively unhindered through the fluid contained in muscle mass.
- The resistance of fat mass is stronger. This resistance is measured.

Using your personal data (sex, weight, age, and height), the scale calculates the following information about your body composition:

- Body fat proportion in %
- Body water in %
- Muscle mass in %
- Bone mass in g
- Weight

The scale has a total of 8 user slots and also offers BMI calculation in the FITvigo app.

Measurement errors

Measurement or calculation errors may arise for the following groups of people during the body analysis:

People who

... spend 10 hours or more per week playing sport,

... are professional athletes, body builders and endurance athletes,

... are over 70 years old,

... are less than 10 years old.

The “normal” weighing function (see next chapter) is not affected by these considerations.

Measuring your weight

WARNING against injuries

- Do not step onto the scale with wet feet or when its surface is damp. Risk of slipping!

- For normal weighing – without body analysis – you do not have to enter any data.
- Weighing is also possible without the FITvigo app connected. If you have connected the FITvigo app, the measured weight will be transmitted to the FITvigo app once the weighing process is finished.

1. Briefly step onto the weighing surface once, using a little force.

“0.0” and the set unit of measurement, e.g. “kg”, will be shown on the display.

2. Position yourself on the scale. Remain standing still and do not hold on to anything. The measured weight will be displayed shortly afterwards.

A few seconds after you step off the weighing surface, the scale will switch off automatically.

When the scale is switched off you can also step onto the scale directly.

Entering personal data

1. Press SET.

A user slot number (P1) will be shown blinking on the display.

2. Using the ◀ or the ▶ button, select the desired user slot.

3. Press SET.

Using the ◀ or the ▶ button, choose your sex:  for male, or  for female. If you are also a very sporty type of person, select the  symbol, too.

4. Press SET.

On the display, the preset value for height (e.g. “175” for cm) will flash.

5. Using the ◀ or the ▶ button, enter your height.

6. Press SET.

The preset value for the age blinks on the display.

7. Using the ◀ or the ▶ button, enter your age.

8. Finally, press SET. You have thus input the necessary data.

“0.0” is shown on the display and you can perform your body analysis. If you do nothing else, the scale will automatically switch off after a few seconds.

- If you hold down the ◀ or the ▶ button, the forward and backward flow of the displayed data will occur more quickly.
- Entered data can only be overwritten by making a new entry.

Automatic user recognition

The scale is equipped with automatic user recognition.

As soon as you step onto the scale, it switches on automatically and within a few seconds shows you the ascertained weight.

If the weight difference between two stored persons is less than 3 kg, automatic user recognition does not work. In this case, "P1-P2" (for example) will be shown on the display:

This indication means that the scale cannot match the person on the scale unambiguously to one of the stored users. The scale hence displays the two users that cannot be differentiated (in this example, "P1" and "P2").

- In such a case, select the correct user slot using the arrow buttons ◀ or ▶.

After selection of the user, the scale automatically resumes the weighing and analysing process.

Body analysis

- Body analysis is only possible if your data has been entered (see previous chapter).
- Body analysis is also possible without the app connected. If you have connected the app, the measured data will be transmitted to the app after the weighing process is finished.
- See the chapter "Weighing Tips".

1. Briefly press SET, in order to switch on the scale.
2. Using the ◀ or the ▶ button, select the user slot in which you have recorded your personal body data.

The data is briefly displayed, then "0.0" and the selected unit of measurement, e.g. "kg".

3. Position yourself barefoot on the scale. Remain standing still and do not hold on to anything.
 - Ensure that each foot is standing completely flat on the weighing surface.
 - There must be no contact between your feet, your legs, your calves, or your thighs. Otherwise, the measurement will be imprecise.

The measured weight will be displayed shortly afterwards. The indicator "o" then runs across the display; the body analysis is being carried out. Remain standing still.

The measured values will then be shown to you repeatedly on the display.

At the same time, in the upper region of the display the type of indicator or value will be shown.

Indicator	Meaning
1. Body fat	Your proportion of body fat in %
2. TBW	Your body water in %
3. Muscle	Your muscle mass in %
4. Bone	Your bone mass in kg
5.	Your body weight in the set unit of measurement

Weighing tips

- Weighing for the body analysis can only be carried out barefoot. In doing so, your feet should only be lightly moistened. Completely dry feet can lead to false results, as the electrical conductivity will be poor then.
- During the measuring process, stand still and do not hold on to anything.
- Do not weigh yourself directly after unusual physical exertion.
- Do not weigh yourself immediately after standing up. Wait approximately fifteen minutes. In this time, the water is able to distribute through your body.
- Always attempt to weigh yourself at the same time, in order to obtain comparable values.

Notes on the ascertained values

The proportion of body fat gives the amount of fat as a percentage of the total weight of the body. A universal recommendation for this value does not exist, since it depends significantly on sex (because of different physiques) and age.

Body fat is vital for day-to-day body function. It protects organs, cushions joints, regulates body temperature, stores vitamins, and serves the body as an energy store.

For women, a body fat proportion between 10% and 13% is essential; for men, between 2% and 5% with the exception of competitive sportsmen.

The proportion of body fat is not an unambiguous indicator of health. Excessive body weight and a too-high proportion of body fat are linked with the appearance of many diseases of civilisation such as diabetes, cardiovascular illnesses, etc. Physical inactivity and poor diet are often triggers for these diseases. There is a clear causal connection between excess weight and physical inactivity.

A considerable part of the human body consists of water. This body water is variably distributed throughout the whole body. Lean muscle tissue contains about 75% water, blood around 83% water, body fat approx. 25% water and bones have a water proportion of about 22%.

- In men approx. 60% of body mass consists of water.
- In women, that value is about 55% (determined by a higher proportion of body fat).

Of this total body water approx. two thirds are stored in the cells and therefore called intracellular water. The other third is extracellular water.

Women ♀					
Age	Body fat				Body water
	Slender	Normal	Slightly overweight	Overweight	Normal
10-16	< 18 %	18-28 %	29-35 %	> 35 %	57-67 %
17-39	< 20 %	20-32 %	33-38 %	> 38 %	47-57 %
40-55	< 23 %	23-35 %	36-41 %	> 41 %	42-52 %
56-85	< 24 %	24-36 %	37-42 %	> 42 %	37-47 %

Men ♂					
Age	Body fat				Body water
	Slender	Normal	Slightly overweight	Overweight	Normal
10-16	< 10 %	10-18 %	19-23 %	> 23 %	58-72 %
17-39	< 12 %	12-20 %	21-25 %	> 25 %	53-67 %
40-55	< 13 %	13-21 %	22-26 %	> 26 %	47-61 %
56-85	< 14 %	14-22 %	23-27 %	> 27 %	42-56 %

For muscle mass there are no universal standard values.

Using the FITvigo app

The data ascertained by the scale are transmitted to the FITvigo app, as long as you have connected it to the scale (see “Connect Device with the App”).

In this regard, pay attention to the following symbols:

Symbol	Meaning
	Existing Bluetooth connection
	Data is being transmitted <ul style="list-style-type: none"> ▪ Data transmission successful: the symbol disappears. ▪ Data transmission unsuccessful: the symbol continues to be displayed. The pending measurement data is temporarily stored in the scale and, after finishing the next measurement, transferred to your smartphone.

Abide by the following points in order to prevent possible transmission problems:

- The distance between the scale and your smartphone should not exceed 10 m.
- There should be no obstructions between the scale and the smartphone.
- Other electronic devices (in particular those that are Bluetooth enabled) should be at least 1 m away from the scale.
- Ensure that the Bluetooth function on your smartphone is switched on.

Cleaning

NOTE on material damage

- Do not immerse the scale in water or other liquids.
- Do not under any circumstances use harsh, scraping, or abrasive cleaning agents or the like to clean the scale. If you do, the surface could be scratched.
- Wipe down the scale as needed using a lightly dampened, soft cloth.
- Disinfect the weighing surface and the sensors from time to time using a suitable disinfecting agent (available at specialist retailers and pharmacies).

Fault/Remedy

Fault	Possible Cause
No function	<ul style="list-style-type: none"> Are the batteries empty or have they been inserted incorrectly?
The scale appears to be displaying false weighing results.	<ul style="list-style-type: none"> Is the scale standing on a firm, even surface? Is a device that can cause electromagnetic disturbances located in the vicinity of the scale? Remove the batteries from the scale and wait a moment. Re-insert the batteries. The data stored up to this point will be retained. For body analysis measurement: Was your body cold during measurement? Then it's possible your blood was circulating poorly. Warm your hands and feet and repeat the measurement.
Analysis of body fat proportion not possible.	<ul style="list-style-type: none"> Were you standing still and barefoot on the weighing surface during the measurement? Your feet must be in complete contact with the weighing surface.
"----" is shown on the display.	<ul style="list-style-type: none"> The maximum capacity of the scale (180 kg) has been exceeded.
"🔋" is shown on the display.	<ul style="list-style-type: none"> The batteries are empty (see "Inserting/ changing batteries").
While attempting to connect the device to the app (pairing), "E I" is shown on the display.	<ul style="list-style-type: none"> Has Bluetooth been activated on your smartphone? Is the FITvigo app active?
Data transmission has failed.	<ul style="list-style-type: none"> Was the distance between the scale and the smartphone too big? Max. distance: 10 m.

Further information about the FITvigo app can be found online at www.fitvigo.de

Do you need help or do you have additional questions?
Then please send us an e-mail to: support@fitvigo.de

Technical Data

Model:	Body Analyser Scale BA 1600 (black) BA 1800 (white)
Battery:	4x R03 (AAA)/1.5 V 
Rated current:	≤ 35 mA
Transmission frequency:	2.4 GHz
Max. Transmission power:	0 dBm
Measuring range:	max. 180 kg
Variance:	0-50 kg = +/- 0.3 kg 51-100 kg = +/- 0.4 kg 101-150 kg = +/- 0.5 kg 151-180 kg = +/- 0.7 kg
Indicator division:	
Bone mass:	100 g
Body water, muscle mass, and body fat:	0.1 %
User slots:	8
Age input:	10-85 years
Height:	100-250 cm
Dimensions of scale:	approx. 320 x 330 x 28 mm
Distributor (no service address):	Waagen-Schmitt GmbH Hammer Steindamm 27-29 22089 Hamburg, Germany

Our products are constantly being refined and improved.
For this reason, design and technical modifications are possible at any time.

Declaration of Conformity

Hereby, Waagen-Schmitt GmbH declares that the radio equipment type body analyser scale BA1600/BA 1800 is in compliance with Directive 2014/53/EU and 2011/65/EU.

The full text of the EU declaration of conformity is available at the following internet address: www.fitvigo.de/support/doc

This declaration loses its validity if modifications are made to the product without our approval.

Hamburg, January 2018
Waagen-Schmitt GmbH
Hammer Steindamm 27-29
22089 Hamburg, Germany

Guarantee

The company Waagen-Schmitt GmbH guarantees for 2 years from the purchase date, the cost-free rectification of defects on the basis of material or fabrication errors through repair or exchange.

In the event of a guarantee claim, please return the product with the proof of purchase (stipulating the reason for claim) to your dealer.

Disposal

Disposing of the packaging

Dispose of the packaging with similar materials. Put cardboard with waste paper, foil with recyclable materials.

Disposing of the product

Dispose of the product in accordance with the regulations that apply in your country.

Devices must not be disposed of in normal household waste.

At the end of its life, the product must be disposed of in an appropriate way. In this way, valuable materials contained in the device will be recycled and the burden on the environment avoided. Hand in the old device at a collection point for electronic waste or at a recycling depot.

For more information, contact your local waste disposal firm or your local administrative body.

Recyclable and disposable batteries do not belong in household waste.

All batteries must be handed in at a collection point in your community, suburb, or at the retailer. Such batteries can thus be disposed of in an environmentally friendly way.

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