BEDIENUNGSANLEITUNG
Operating manual
Mode d'emploi
Istruzioni d'uso

ADE

Mechanische Personenwaage
Mechanical floor scales
Du pèse- personne mécanique
Bilancia meccanica pesapersona

M309800

M309800-210705-Rev003-UM-de-e-fr-it

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www.ade-germany.de

General

This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

Your mechanical scale made by ADE is a precision instrument and must be treated accordingly. Please follow the instructions below in order to obtain the best results at all times.

Contents of box

- Mechanical round-dial scale
- Operating manual

Safety

- Handle your scale with care - it is a precision instrument.
- Use the scale on a firm and level surface
- Operate the unit only within the permissible ambient conditions.
- Do not operate the unit in hazardous or unstable environments.
- Clean with a dry cloth, do not use harsh cleaning agents.
- Do not immerse the appliance in water or other liquids.

Installation and Use

Zero setting

Before you can use the scale for the first time, you must adjust the zero point manually.

- Place the scale on a flat and solid surface.
- After transport and setting up, the movable parts of the scale must first move to their respective initial positions. You can accomplish that by operating the scale and stepping onto the platform and down again several times.
- For setting the zero point, however, the platform must be clear.
- Adjust the scale by turning the dial above the display to the left or right until the pointer on the dial points exactly to zero.
- The scale is ready for operation now.

You should repeat the procedure from time to time so you can be sure that you always get the best results. Proceed as described above. This is particularly important whenever you move the instrument to a new place / different surface.

Operating the scale

- Step onto the platform of the scale.
- Be sure to place your feet in the centre of the platform. The distances between your feet and the edges of the platform should be more or less the same on all sides. Try not to move.
- Wait until the pointer stops moving.

The pointer will always tremble a little because it is impossible to stand absolutely motionless on the platform. However, the error is minimal. You may be confident to have a precise enough result at all times.

Note: To get reliable statistical weighing results take your weight every day at the same time.

Cleaning and maintenance

Cleaning and Storage

- For cleaning just take a damp cloth or common disinfection products.
- Do not use any aggressive cleaning fluids.
- In order to achieve a long lifetime of your scale, make sure no water or fluids can penetrate and keep the scale away from high humidity or direct heat or sunlight.

Warranty – Liability

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

Any legally guided customer rights are not affected by this warranty agreement.

Technical data

- Max. capacity: 150 kg
- Graduation: 500 g
- Dimensions: 320 x 70 x 435 mm
- Weight: abt.: 2,8 kg
- Weighing platform: Metal
- Easy to read
- Non slip vinyl mat

Declaration of Conformity

Manufacturers Declaration of Conformity

This product were manufactured in accordance with the harmonised European Standards.

It corresponds to the regulations of below listed EC-directive:

- Machinery 2006/42/EC
as appropriate amended.

This declaration loses its validity, in case modifications on the product are made without manufacture's approval.

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ADE Germany GmbH