
Elektronische Veterinärwaage
Electronic veterinary scale
Balance vétérinaire électronique
Bilancia elettronica per uso veterinario
Báscula Veterinaria Electrónica

SERIE/SERIES/SÉRIE MV304600

Betriebsanleitung
User manual
Notice d'utilisation
Manuale d'istruzioni
Manual de instrucciones

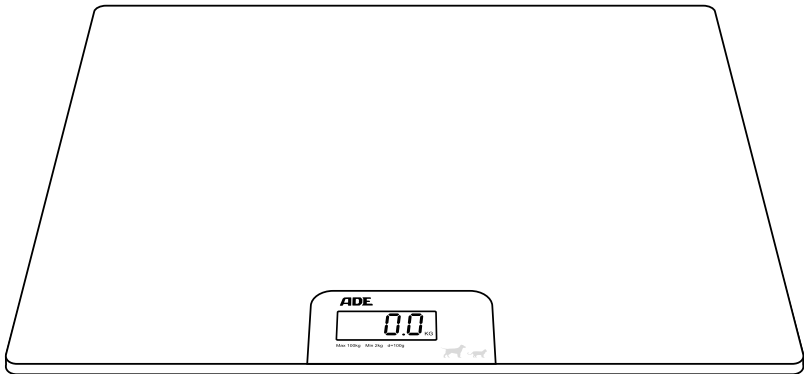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
Table of contents

| | |
|--|-----------|
| Package contents | 26 |
| General information | 27 |
| Reading and storing the user manual | 27 |
| Proper use..... | 27 |
| Explanation of symbols | 27 |
| Safety | 28 |
| Explanation of notes | 28 |
| General safety notes | 28 |
| Battery notes..... | 31 |
| Description of the device | 34 |
| Symbol description | 35 |
| First use | 35 |
| Checking the scale and package contents..... | 35 |
| Setting up the scale..... | 36 |
| Inserting the batteries..... | 36 |
| Use | 37 |
| Switching on..... | 37 |
| Switching off | 37 |
| Selecting the weighing unit..... | 38 |
| Weighing an animal | 38 |
| Troubleshooting | 39 |
| Cleaning and maintenance | 40 |
| Cleaning | 40 |
| Maintenance | 41 |
| Transport..... | 41 |
| Storage | 42 |
| Technical data | 42 |
| Disposal | 43 |
| Disposing of the packaging | 43 |
| Disposing of the appliance..... | 43 |
| Declaration of Conformity | 44 |
| Warranty | 44 |

Package contents

ENGLISH

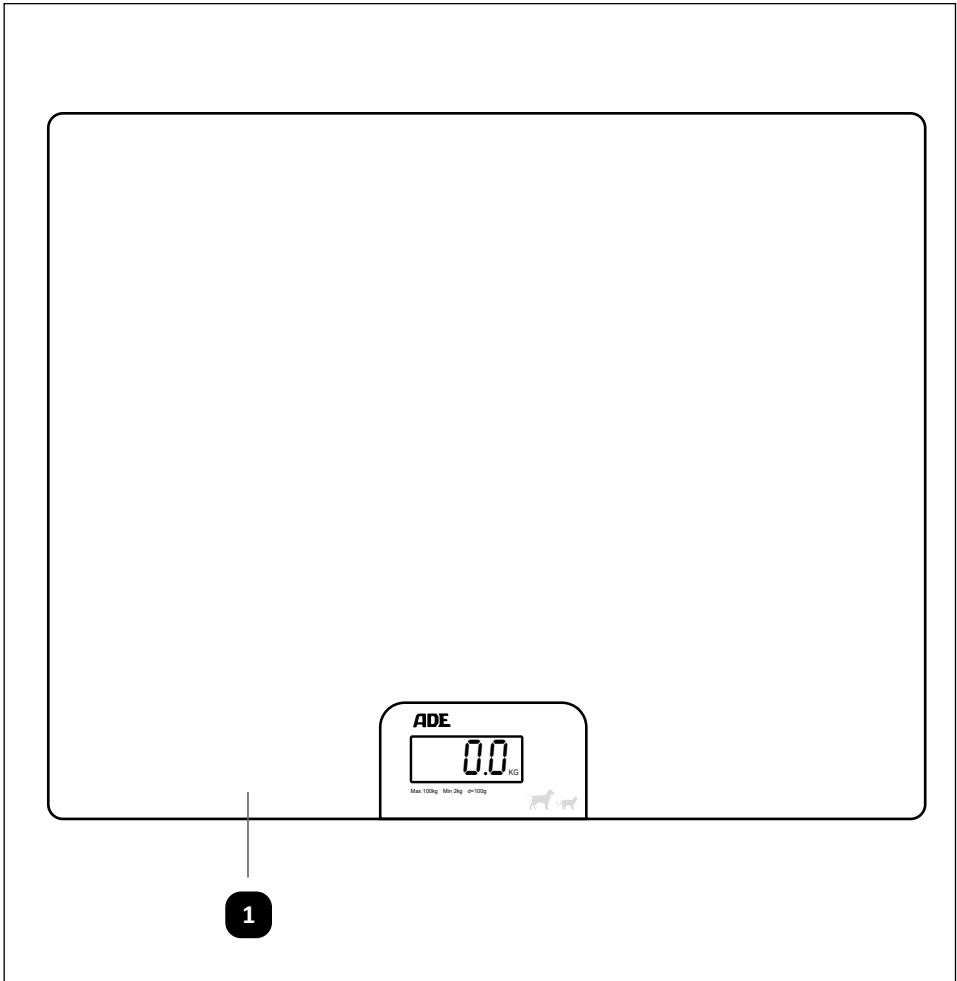Package contents

- 1** Scale
- 2** User manual (not pictured)
- 3** Batteries (AAA), 2× (not pictured)

General information

Reading and storing the user manual

This user manual belongs to the MV304600 electronic veterinary scale (hereinafter referred to as the “scale”). It contains important information about initial use and handling.

Read the user manual carefully, especially the safety instructions, before you begin using the scale. Failure to heed this user manual may cause severe injury or damage to the appliance.

The user manual is based on the standards and rules in force in the European Union. When abroad, you must also observe country-specific guidelines and laws.

Store the user manual for future use. Should you give the scale to another person, it is vital that you also pass on this user manual.

Proper use

The scale is exclusively intended for weighing pets and small and medium-sized animals. The maximum weight of 100 kg must not be exceeded. It is intended for private and commercial use.

The scale is intended for use as a non-automatic scale. This means that animals must be manually placed in the centre of the weighing platform. When the weighing value has been reached, the weight can be read off.

Only use the scale as described in this user manual. Any other use is deemed improper and may result in damage to property or even personal injury. The manufacturer or vendor accepts no liability for damage caused by improper or incorrect use.

Explanation of symbols

The following symbols can be found in this user manual, on the appliance or on the packaging.

This symbol provides you with useful supplementary information on setup and handling.

Read and observe the user manual. This symbol indicates that the user must read the user manual before the first use of the appliance.

CE symbol

Direct current

Safety

Explanation of notes

The following symbols and signal words can be found in this user manual, on the appliance or on the packaging.

WARNING!

This signal symbol/word designates a hazard with a moderate degree of risk which may lead to death or severe injury if not avoided.

CAUTION!

This signal symbol/word designates a hazard with a low degree of risk which may lead to minor or moderate injury if not avoided.

NOTICE!

This signal word warns against potential damage to property.

General safety notes

WARNING!

Danger for children and persons with impaired physical, sensory or mental capacities (e.g. partially disabled persons, older persons with reduced physical and mental capacities) or lack of experience and knowledge (e.g. older children).

- The scale can be used by children 8 years of age and above and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning the use of the scale in a safe way and if they understand the hazards involved. Children are not allowed to play with the scale. Cleaning and user maintenance must not be performed by unsupervised children.
- Keep children under 8 years of age away from the scale.
- Children shall not play with the scale.

- Do not leave the scale unattended while in operation.
- Do not let children play with the packaging wrapper. They may get caught in it when playing and suffocate.

CAUTION!

Risk of injury!

Improper handling of the appliance may cause injury.

- Only use the scale when it is fully and correctly assembled.
- Only use the supplied accessories.
- Never use the scale near highly flammable or explosive substances.
- Do not use the scale if it exhibits visible signs of damage.
- Do not open the housing; have repairs carried out by a qualified professional. Contact a qualified workshop for this.
- Only parts that comply with the original device specifications may be used for repairs. This scale contains electrical and mechanical parts that are essential for protection against hazards.
- Do not attempt to repair or modify the scale single-handedly. This can affect the operational safety of the scale. Independently performed repairs, installation of spare parts not supplied by the manufacturer, improper connection or operation of the scale will render the warranty null and void.
- Do not immerse the scale in water or other liquids.
- Ensure that children do not insert objects into the scale.
- Always switch off the scale and remove the batteries when you are not using the scale, are cleaning, performing maintenance, replacing parts or in the event of a fault.

NOTICE!

Risk of damage!

Improper handling of the scale may result in damage to the scale.

- Only use approved accessories and peripherals.
- Handle the scale with due care. Note that the scale is a precision instrument.
- Observe the maximum weighing capacity of the scale. Do not exceed it.
- Do not expose the weighing platform to jolts and impacts.
- The scale must be operated as far away as possible from other equipment or sources of electromagnetic or other interferences (e.g. powerful motors or mobile HF communications equipment), as these can affect the measurement results.
- Only use the scale in dry indoor areas. It is not intended for outdoor use.
- Only use the scale within the permitted ambient conditions.
- Never place the scale near hot surfaces.
- Use the scale at constant ambient temperatures and avoid using it in draughty places as this may yield incorrect measurement results.
- Never expose the scale to high temperatures (e.g. heat, direct sunlight), moisture, humidity or weather conditions (e.g. rain).
- Never fill the scale with liquid.
- Never attempt to clean the scale by submerging it in water, and do not use a steam cleaner to clean it. The scale may otherwise get damaged.
- Do not put the scale in a dishwasher. It would be destroyed as a result.

- Discontinue use of the scale if its plastic components have any cracks, flaws or deformations. Only replace damaged components with suitable original spare parts.
- Store the scale in a dry, cool place.

Battery notes

WARNING!

Risk of injury!

Mishandling of the batteries may cause gas leakage, electrolyte (battery fluid) leakage, fire or explosion.

- Ensure the correct polarity when inserting the batteries.
- If necessary, clean the battery and device contacts before inserting the batteries.
- Do not short circuit the batteries.
- Do not recharge the batteries. Attempting to charge non-rechargeable batteries can lead to overheating and cause a fire.
- Store unpacked batteries in such a way that the battery contacts do not touch in order to avoid short circuiting. Ideally, keep the batteries in their original packaging.
- Keep unpacked batteries away from metal objects to avoid short circuits.
- Do not take the batteries apart.
- Never attempt to modify the batteries.
- Do not crush, deform or make any holes in the batteries.
- Do not throw the batteries into an open fire.
- Do not heat the batteries or carry out any welding or soldering work on them.
- Do not expose batteries to unrestricted sunshine or heat.
- Do not allow the batteries to come into contact with water.

- Do not use the batteries if they are damaged.
- Avoid deep-discharging the batteries.
- Discontinue using of the scale if it is damaged or if the battery cover is bulging.
- Store batteries out of the reach of children. If you suspect that a battery has been swallowed or has otherwise entered the body, consult a physician immediately.
- Do not allow children to change the batteries without supervision.
- Always change the entire set of batteries. Never mix old and new batteries or batteries with different charges. Replace all batteries with new batteries of the same type and brand.
- If you are no longer able to securely close the battery compartment in which the batteries are located, do not use the scale. Remove the batteries and store them in a location which is inaccessible to children.

CAUTION!

Risk of injury!

In case of malfunction or damage, electrolyte (battery fluid) may leak from a battery. Contact with fluid leaking from a battery can lead to burns.

- Do not handle leaking batteries without protective gloves.
- Avoid contact with leaked battery fluid.
- In the event of contact with leaked battery fluid, immediately rinse the area with clean water.
- Immediately consult a doctor if leaked battery fluid comes into contact with the eyes.

**NOTICE!****Risk of damage!**

If discharged batteries remain in the scale for too long, the electrolyte may leak out, causing damage to the scale.

- Rechargeable batteries should be charged regularly.

**NOTICE!****Risk of damage!**



Improper storage can cause damage to the batteries.

- Remove batteries from the scale if it is not going to be used for an extended period of time.
- Store the batteries in a cool, dry place.

Description of the device

The scale is exclusively intended for weighing pets and small and medium-sized animals. The maximum weight of 100 kg must not be exceeded.

The scale is intended for use as a non-automatic scale. This means that animals must be manually placed in the centre of the weighing platform. When the weighing value has been reached, the weight can be read off. The batteries are part of the supplied contents.

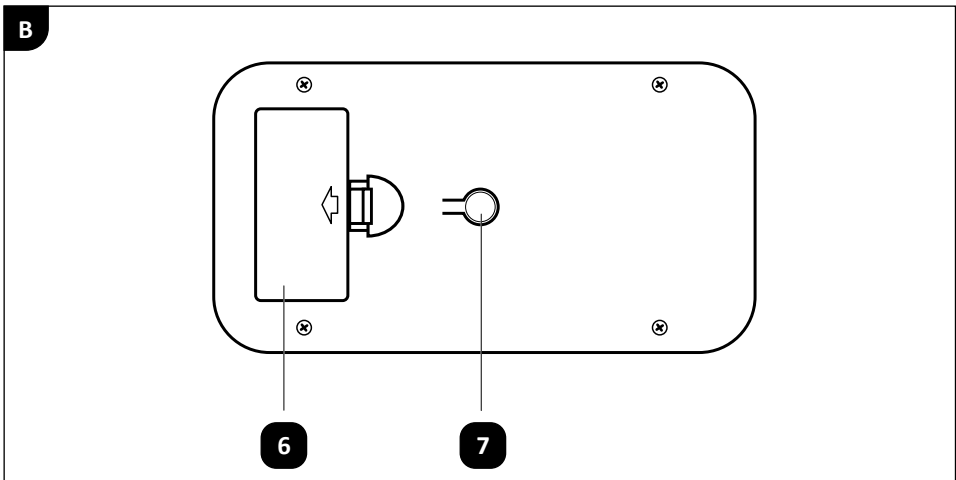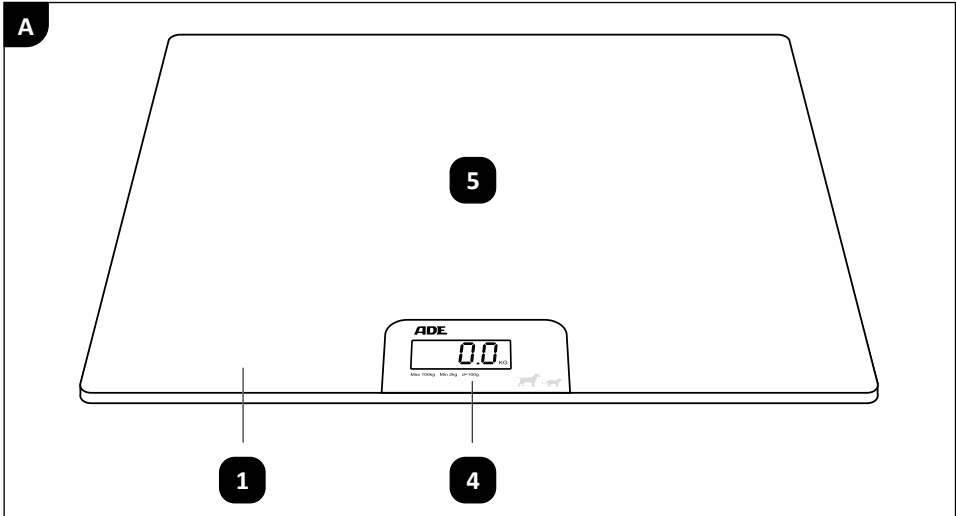1 Scale**6** Battery compartment cover**4** Display**7** "Weighing unit" button**5** Weighing platform

Symbol description

| Symbol | Meaning |
|--------|------------------------|
| KG | Kilogram weighing unit |
| LB | Pound weighing unit |

First use

Checking the scale and package contents

Risk of damage!

If you are not cautious when opening the packaging with a sharp knife or other pointed objects, you may quickly damage the scale.

— Be very careful when opening the packaging.

1. Remove the scale **1** from the packaging.
2. Check to make sure that all parts have been included in the delivery (see **page 26**).
3. Check whether the scale or the individual parts exhibit signs of damage. If this is the case, do not use the scale. Contact the distributor at the address stated on the back cover of this user manual.
4. Remove all packaging materials, protective films and labels attached to the appliance for transport protection or advertising purposes.

Keep the original packaging and packaging materials for future transport of the scale.

Setting up the scale

NOTICE!

Risk of damage!

The scale can get damaged due to overload.

- Do not place excessive pressure on the scale.
- Place the scale **1** on a level surface. Using the scale on a tilted surface can lead to incorrect measurement results.

Inserting the batteries


WARNING!


Risk of explosion or fire!

Incorrect installation of batteries can lead to a short circuit or overload, causing a rapid battery temperature rise. Battery overheating may lead to gas or battery fluid leakage or explosion, which, in turn, may result in injury.


- Make sure that the polarity is correct when inserting the batteries.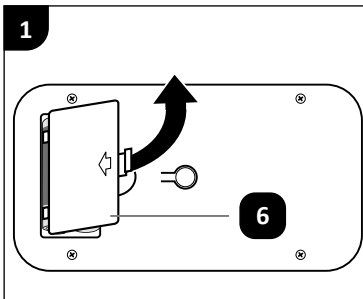

1. Open the battery compartment. Press the lock of the battery compartment cover **6** slightly to the side (see **Fig. 1**).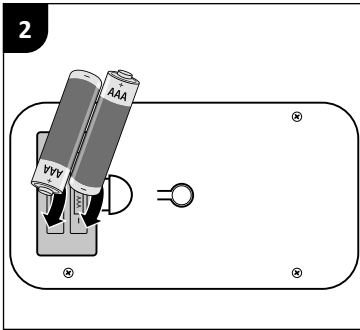

2. Insert the batteries **3** in the battery compartment (see **Fig. 2**).

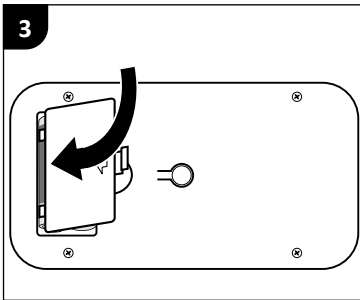3. Close the battery compartment. Place the battery compartment cover on the battery compartment and press the battery compartment cover lock in slightly until it clicks into place (see **Fig. 3**).

Use

Switching on


Before turning on the scale, make sure there are no animals on the weighing platform. If there is an animal on the weighing platform when you turn on the scale, the scale will not be ready for use. Remove the animal from the scale, turn off the scale and repeat the switch-on process.

- To switch on the scale **1**, tap your foot on the weighing platform **5**.
“8888” will appear on the display **4**. Then the display will show “0.0”.

Switching off

The scale has an automatic switch-off function. There is no need for the user to switch off the scale.

Selecting the weighing unit

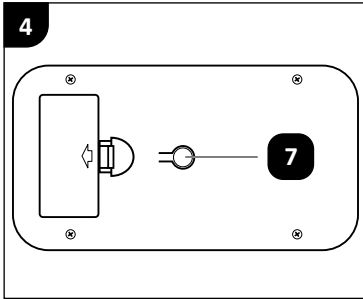1. Switch on the scale **1**.
“8888” will appear on the display **4**. Then the display will show “0.0”.
2. To select the desired weighing unit (KG/LB), press the “weighing unit” button **7** until the desired weighing unit (KG/LB) is displayed.

Weighing an animal

NOTICE!

Risk of damage!

Improper handling of the scale may result in damage to the scale.

- Do not overload the scale. Do not exceed the maximum weighing capacity.
- When the scale is overloaded, the error message “Err” is displayed. In this case, remove the load from the weighing platform.

Carefully put the animal in the centre of the weighing platform.
Never exceed the maximum weighing capacity of the scale (see chapter „Technical data“ on page 42).

1. To switch on the scale **1**, tap your foot on the weighing platform **5**.
2. Wait a few seconds for the scale to return to zero.
“8888” will appear on the display **4**. Then the display will show “0.0”.
The scale is ready for use.
3. Put the animal on the weighing platform and make sure that it stays as still as possible.
The result first flashes twice on the display and is then displayed for 10 seconds.
The weight of the animal can now be read off the display.

Troubleshooting

| Problem | Possible causes | Solution |
|---------------------------------------|--|---|
| The scale 1 does not work. | The scale is not switched on. | Switch on the scale. |
| | The batteries 3 are depleted. | Insert new batteries. |
| | You did not tap your foot firmly enough on the weighing platform 5 . | Tap your foot more firmly on the weighing platform. |
| The display 4 is blank. | The scale has switched off. | Switch on the scale again. |
| The weight display is not stable. | The scale is exposed to vibrations. | Avoid vibrations or relocate the scale. |
| | The scale is disturbed by electromagnetic fields from other equipment or static charges. | Turn off the equipment near the scale or relocate the scale. |
| The weighing result is clearly wrong. | The scale is not on a level surface. | Place the scale on a level surface. |
| | The scale is exposed to temperature conditions that are outside the permissible range. | Only operate the scale within the permissible temperature conditions. |
| | The scale is disturbed by electromagnetic fields from other equipment or static charges. | Turn off the equipment near the scale or relocate the scale. |
| The display shows "Err". | The weight on the scale exceeds the maximum weighing capacity of 100 kg. | Remove the item from the scale. Restart the scale if necessary. |
| The display shows "Lo". | The batteries are depleted. | Insert new batteries. |

Cleaning and maintenance

Risk of short circuit!

Water or other liquids that have penetrated the housing may cause a short circuit.

- Never immerse the scale in water or other liquids.
- Make sure that no water or any other liquids get into the housing.

Risk of damage!

Improper handling of the scale may cause damage to it.

- Do not use any aggressive cleaners, brushes with metal or nylon bristles, or sharp or metallic cleaning utensils such as knives, hard scrapers, etc. They could damage the surfaces.
- Do not put the scale in a dishwasher. It would be destroyed as a result.

Cleaning

1. Use a slightly damp cloth or commercially available mild disinfectant to clean all surfaces.
2. Let all parts dry completely.

Maintenance

Changing the batteries

WARNING!

Risk of explosion or fire!

Incorrect installation of batteries can lead to a short circuit or overload, causing a rapid battery temperature rise. Battery overheating may lead to gas or battery fluid leakage or explosion, which, in turn, may result in injury.

- Make sure that the polarity is correct when inserting the batteries.

NOTICE!

Risk of damage!

The scale can get damaged due to overload.

- Do not place excessive pressure on the scale.

To change the batteries, refer to the illustrations in the chapter “Inserting the batteries”.

1. Open the battery compartment. Press the lock of the battery compartment cover **6** slightly to the side.
2. Remove the depleted batteries **3**.
3. Insert new batteries in the battery compartment.
4. Close the battery compartment. Place the battery compartment cover on the battery compartment and press the battery compartment cover lock in slightly until it clicks into place.

Transport

- Use the original packaging for the scale **1**. Use the packaging material so that the scale is protected and secured in the packaging during transport.

Storage

All parts must be completely dry before being stored.

- Always store the scale in a dry place.
- Protect the scale from direct sunlight.
- The scale must be stored in a securely closed, dry and frost-free place unreachable for children at a temperature between +5 °C and +40 °C.

Technical data

Scale

| | |
|------------------------------|---------------------|
| Model: | MV304600 |
| Maximum weighing capacity: | 100 kg |
| Minimum weighing capacity: | 2 kg |
| Digit increment: | 0.1 kg |
| Dimensions: | 800 × 600 × 30.3 mm |
| Weight: | approx. 5.5 kg |
| Operating temperature range: | +5 °C to +40 °C |

Batteries

| | |
|-------|-----|
| Type: | AAA |
|-------|-----|

Disposal

Disposing of the packaging

Dispose of the packaging separated into single type materials. Dispose of paperboard and cardboard with waste paper and plastics with recyclable waste.

Disposing of the appliance

(Applicable in the European Union and other European states with systems for the separate collection of reusable waste materials)

Old devices must not be disposed of in the household waste! If the product can no longer be used, every user is required by law to dispose of the device separately from household waste, e.g. at a collection point in their community/borough. This ensures that old devices are recycled properly and also avoids any negative impact on the environment. For this reason, electrical equipment is marked with the symbol shown here.

Batteries and rechargeable batteries may not be disposed of with household waste! As the end user you are required by law to bring all rechargeable and non-rechargeable batteries, regardless of whether they contain harmful substances* or not, to a collection point run by the communal authority or borough or to a retailer, to allow them to be disposed of in an environmentally friendly manner.

Turn in the batteries at your collection point in a discharged state only!

*labelled with: Cd = cadmium, Hg = mercury, Pb = lead

Declaration of Conformity

Appliances bearing the CE mark meet all applicable Community regulations for the European Economic Area.

The EU declaration of conformity can be requested from the address specified on the back of the user manual.

Warranty

ADE Germany GmbH will repair or replace this product provided proof is furnished that damage caused by defects in materials and workmanship occurred within one year from the date of purchase. Proof of purchase is required.

The warranty does not apply to all moving parts, such as rechargeable batteries or cables. Using accessories other than those approved by ADE Germany GmbH will render the warranty null and void. This applies in particular to the use of non-approved rechargeable batteries.

The warranty does not cover items damaged by normal wear and tear or damage caused by accident or misuse. The warranty does not cover appliances that have been tampered by unauthorised persons. This agreement does not invalidate the customer's statutory rights.

For any complaints, customer service or provision of spare parts, please contact the dealer from whom you purchased the scale.

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© Copyright