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 info@ade-germany.de

 www.ade-germany.de

This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

This operating manual describes how to safely use and care for your products. Keep this operating manual in a safe place in case you would like to refer to it later. If you pass this item on to someone else, pass on this manual as well.

Failure to observe the recommendations in this operating manual can lead to injuries or damage to your product.

Safety

- This item can be used by children 8 years of age and above, as well as by persons with reduced physical, sensory, or mental capacities or lacking experience and knowledge, if they are supervised or have been instructed in the safe use of the item and understand the dangers that may arise.
- Children must not play with the item.
- Cleaning and user maintenance must not be carried out by children, unless they are 8 years of age or older and are being supervised.
- Batteries, when swallowed, can be life-threatening. You should therefore keep the batteries and scales out of reach of small children. If a battery is swallowed, medical assistance must be sought immediately.

DANGER for Children

- Keep children away from packaging material. Risk of choking if swallowed. Also pay attention that children do not pull the packaging material over their heads.

RISK of fire/burning and/or explosion

- There is a risk of explosion if batteries are replaced incorrectly. Replace the battery using only the same or an equivalent type of battery.
- Batteries must not be recharged, reactivated by other means, dismantled, thrown into fire, or short-circuited. Otherwise there is an increased risk of explosion and of leakage; furthermore, gases may escape.
- Avoid contact with skin, eyes, and mucosa. If you come into contact with battery acid, flush the affected site immediately with plenty of clean water and see a doctor promptly.

BEWARE of material damage

- Place the scales on a stable, even surface.
- Do not overload the scales. Place animals on the scales exclusively for weighing purposes.
- Protect the scales from sharp jolts and vibrations.
- Remove the batteries from the scales if they have been used up or if you are not going to use the scales for a long time. You will thus avoid damage that may result from the leakage of battery acid.
- When inserting and changing batteries, pay attention to the correct polarity (+/-).
- Do not subject the battery to any extreme conditions, by – for example – storing it on radiators or under direct sunlight. Increased risk of leakage!
- Clean the battery and device contacts, as needed, before inserting the battery.
- Only insert batteries of the same type; do not use different types or used and new batteries with each other.

- Do not make any modifications to the item. Only allow repairs to be carried out at a specialist workshop. Repairs carried out incorrectly can considerably endanger the user.
- Do not place the scales in the vicinity of devices that may emit electromagnetic radiation, such as mobile phones, wireless devices, or the like. Doing so may lead to incorrect display indicators or malfunctions.

Scope of delivery

- Veterinary Scale, completely assembled
- Operating Manual
- A lithium long-life battery (Cr2032)

Intended Use

The veterinary scale is intended for weighing pets and small to midsized animals. The maximum carrying capacity of 100 kg must not be exceeded.

Start-Up

The balance is powered by a long-life lithium battery (Cr2032). Before using the balance for the first time, remove the insulating strips from the battery.

Insert and replace the battery:

In case the display shows 'LO' the battery is weak and needs replacement. Use a Cr2032 (lithium) battery.

Open the battery compartment underneath the scale. Carefully remove the battery by gently pushing the black plastic strap to the side. Carefully remove the battery to the side so that the metal tab is not bent. When inserting the new battery, make sure that it is inserted according to the pole marking.

Positioning the scales

Place the scales on an even, firm, and dry surface. Do not place the scales on a carpet. The scale and base surface, e.g. a table, must not wobble in any case. An uneven surface could result in inaccurate readings.

Weighing

1. Place and use the scale on a safe and even ground (no carpet).
2. To start the scale tap on the platform.
3. Wait for some seconds until the display comes to zero. A short function test '8888' can be seen on the display before it comes to '0.0'.
4. The scale is now ready for weighing.
5. Place or set the animal on the platform and make sure the animal is not moving too much.
6. The result is indicated in the display, blinks 2 times and is hold for around 10 seconds.

Switching the scale off

The scale is equipped with an automatic switch-off function. It is not necessary for the user to switch off the scale.

Choosing the unit of measurement

Underneath the scale you can manually choose the weighing unit by shifting the sliding button to either kg, lb or st.

Taking the weight the scale will now indicate the result in the selected mode.

Care and maintenance

Cleaning

- For cleaning just use a damp cloth or common disinfection fluids.
- Do not use any aggressive cleaning fluids.
- Make sure no water or cleaning fluids is penetrating the scale.
- Do not under any circumstances use harsh, scraping, or abrasive cleaning agents or the like to clean the scales. If you do, the surface could be scratched.

Fault/Remedy

Fault	Possible Cause
No function	Battery empty or inserted the wrong way around?
The scales appear to be displaying false weighing results	Are the scales standing on a firm, even surface? Are the scales exposed to extraordinary temperatures? Is a device that can cause electromagnetic disturbances located in the vicinity of the scales? Remove the battery from the scales and wait a moment. Reinsert the battery.
,ERR' is shown on the display	The maximum capacity of the scales (100 kg) has been exceeded.
,LO' is shown on the display	The batteries are weak and need replacement.

Storage and Transport

Please keep all original parts and pieces for an eventual sending back of the scale in order to avoid any possible damage during transport which automatically leads to non warranty.

Disposal

Waste electrical equipment do not belong in household waste. Devices with this marking may not be disposed of as residual waste, but must be recycled.

Remove (if possible) all batteries and rechargeable batteries from the devices and send them to the battery disposal.

Make sure that only depleted batteries or batteries with insulated poles are disposed of so that there is no short circuit!

Warranty – Liability

ADE will repair or replace this product within a period of 1 year from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

Technical data

Model:	MV302600
Max. Capacity:	100 kg
Graduation:	100 g
Display:	easy to read 25 mm LC Display
Dimensions:	620 x 45 x 830 mm
Housing material:	Wood
Power supply:	1 x Cr2032 Lithium Battery
Zero setting:	Automatically
Weight:	Approx. 5,5 kg
Operating temperature:	10 to 40°C

Declaration of Conformity

Declaration of conformity by the manufacturer

This product has been manufactured in accordance with the harmonised European standards. They conform to the regulations of the EC-directives listed below:

- EMC Directive 2014/30/EU
 - RoHS Directive 2011/65/EU
- in their respectively applicable versions.

This declaration will lose its validity if any modification is made to the scale without our consent.

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ADE Germany GmbH