Bedienungsanleitung
Operating Manual
Mode d'emploi
Manual de funcionamiento
Manuale operativo

ADE

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info@ade-germany.de

www.ade-germany.de

This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

This operating manual describes how to safely use and care for your products. Keep this operating manual in a safe place in case you would like to refer to it later. If you pass this item on to someone else, pass on this manual as well.

Failure to observe the recommendations in this operating manual can lead to injuries or damage to your product.

Safety

- This item can be used by children 8 years of age and above, as well as by persons with reduced physical, sensory, or mental capacities or lacking experience and knowledge, if they are supervised or have been instructed in the safe use of the item and understand the dangers that may arise.
- Children must not play with the item.
- Cleaning and user maintenance must not be carried out by children, unless they are 8 years of age or older and are being supervised.
- Batteries, when swallowed, can be life-threatening. You should therefore keep the batteries and scales out of reach of small children. If a battery is swallowed, medical assistance must be sought immediately.

DANGER for Children

- Keep children away from packaging material. Risk of choking if swallowed. Also pay attention that children do not pull the packaging material over their heads.

RISK of fire/burning and/or explosion

- There is a risk of explosion if batteries are replaced incorrectly. Replace the battery using only the same or an equivalent type of battery.
- Batteries must not be recharged, reactivated by other means, dismantled, thrown into fire, or short-circuited. Otherwise there is an increased risk of explosion and of leakage; furthermore, gases may escape.
- Avoid contact with skin, eyes, and mucosa. If you come into contact with battery acid, flush the affected site immediately with plenty of clean water and see a doctor promptly.

BEWARE of material damage

- Place the scales on a stable, even surface.
- Do not overload the scales. Place animals on the scales exclusively for weighing purposes.
- Protect the scales from sharp jolts and vibrations.
- Remove the batteries from the scales if they have been used up or if you are not going to use the scales for a long time. You will thus avoid damage that may result from the leakage of battery acid.
- When inserting and changing batteries, pay attention to the correct polarity (+/-).
- Do not subject the battery to any extreme conditions, by – for example – storing it on radiators or under direct sunlight. Increased risk of leakage!
- Clean the battery and device contacts, as needed, before inserting the battery.
- Only insert batteries of the same type; do not use different types or used and new batteries with each other.

- Do not make any modifications to the item. Only allow repairs to be carried out at a specialist workshop. Repairs carried out incorrectly can considerably endanger the user.
- Do not place the scales in the vicinity of devices that may emit electromagnetic radiation, such as mobile phones, wireless devices, or the like. Doing so may lead to incorrect display indicators or malfunctions.
- There are feet on the underside of the scales. Since surfaces are made from various materials and are treated with diverse care products, the possibility cannot be completely excluded that some of these substances contain constituents that attack and soften the feet of the device. If necessary, place an anti-slip underlay beneath the scales.

At a glance

1	Weighing surface	
2	Display	
3	ON/OFF-Button	Switch the scales on and off; Set the values to zero (tare function)
4	UNIT button	Changing the unit of measurement
5	HOLD button	De-/Activate the HOLD-Function
6	Battery compartment	(bottom side)

Scope of delivery

- Veterinary Scale, completely assembled
- Operating Manual
- **Not included:** 2 x AAA batteries

Intended Use

The veterinary scale is intended for weighing pets and small to midsized animals. The maximum carrying capacity of 100 kg must not be exceeded.

Start-Up

Insert and replace the battery:

To install batteries, remove the battery compartment cover located on the bottom of the scale. Insert two AAA alkaline batteries. Make sure that they are inserted according to the pole marking.

When battery power is low, the display will show 'lo'. Please replace the batteries.

Positioning the scales

Place the scales on an even, firm, and dry surface. Do not place the scales on a carpet. The scale and base surface, e.g. a table, must not wobble in any case. An uneven surface could result in inaccurate readings.

Weighing

1. Press the ON/OFF button.
2. Wait for a couple of seconds until the display shows zeros. A short function test '8888' can be seen on the display before it comes to '0.00' or 0: 0. Once the scale shows zeros on the display, it is ready to use.
3. Place your pet on the scale, as close to the centre of the scale as possible. The weight reading will show on the display.

Switching the scale off

To switch the scale off, press ON/OFF button.

The scales automatically switch off if they are not used for approx. 60 seconds.

Hold function

The Hold function allows you obtain an accurate weight reading, even if the animal is moving on the scale. With the animal on the scale, press HOLD once.

The weight reading will flash several times, and then display the correct weight for around six seconds. Afterwards, the scale switches off automatically.

Tare function

The Tare function allows you to remove unwanted weight readings, such as a towel or blanket.

With the scale switched off, place the item you wish to tare off (towel or blanket) onto the platform.

Press ON/OFF once. The display will show zero.

You can now place your pet on the scale and only the weight of pet will display.

When switching the scale off the tare weight will be automatically cancelled.

Choosing the unit of measurement

At any time, with the scale switched on press UNIT to toggle between kg and lb readings.

The scale will

start up in the last setting.

Care and maintenance

Cleaning

- For cleaning just use a damp cloth or common disinfection fluids.
- Do not use any aggressive cleaning fluids.
- Make sure no water or cleaning fluids is penetrating the scale.
- Do not under any circumstances use harsh, scraping, or abrasive cleaning agents or the like to clean the scales. If you do, the surface could be scratched.

Fault/Remedy

Fault	Possible Cause
No function	Battery empty or inserted the wrong way around?
The scales appear to be displaying false weighing results	Are the scales standing on a firm, even surface? Are the scales exposed to extraordinary temperatures? Is a device that can cause electromagnetic disturbances located in the vicinity of the scales? Remove the battery from the scales and wait a moment. Reinsert the battery.
,ERR' is shown on the display	The maximum capacity of the scales (100 kg) has been exceeded.
,LO' is shown on the display	The batteries are weak and need replacement.

Storage and Transport

Please keep all original parts and pieces for an eventual sending back of the scale in order to avoid any possible damage during transport which automatically leads to non warranty.

Disposal

Waste electrical equipment do not belong in household waste. Devices with this marking may not be disposed of as residual waste, but must be recycled.

Remove (if possible) all batteries and rechargeable batteries from the devices and send them to the battery disposal.

Make sure that only depleted batteries or batteries with insulated poles are disposed of so that there is no short circuit!

Warranty – Liability

ADE will repair or replace this product within a period of 1 year from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

Technical data

Model:	MV303600
Max. Capacity:	100 kg
Graduation:	50 g
Display:	LCD display with four active digits
Dimensions:	716mm x 40 mm x 412mm
Housing material:	Safety glass
Power supply:	2 x AAA batteries
Zero setting:	Automatically
Weight:	8.0 kg
Operating temperature:	10 to 40°C
Function Keys	ON/OFF, UNIT, HOLD

Declaration of Conformity

Declaration of conformity by the manufacturer

This product has been manufactured in accordance with the harmonised European standards. They conform to the regulations of the EC-directives listed below:

- EMC Directive 2014/30/EU

- RoHS Directive 2011/65/EU

in their respectively applicable versions.

This declaration will lose its validity if any modification is made to the scale without our consent.

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ADE Germany GmbH