
Gebrauchsanweisung
Operating instructions
Istruzioni d'uso
Instrucciones de uso
Mode d'emploi

Körperlängen-Messband
Body Measuring Tape
Altimetro a nastro
Cinta Métrica de pared
Micro toise

ADE
MZ10017

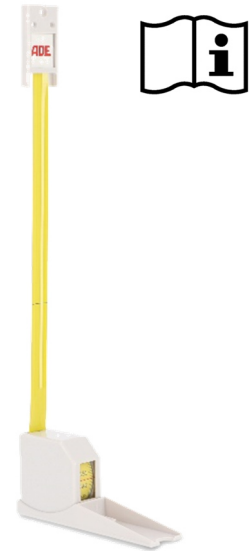MZ10017-220126-Rev006-de-e-it-es-fr

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- Make sure that there are no power, water or other supply lines in the area of the drill holes. Scope of delivery: - Body Measuring Tape - Mounting material (3 screws, 3 dowels) - Operating instructions Assembly ①: - In order to find the correct fixing point, place the Body measuring tape on the ground next to the wall (A). - Pull out the wall mounting bracket until the reading mark reaches exactly 'zero'. - Use the mounting material to fix the wall bracket on the wall and re-check position (B). - Tighten the screws and make sure the wall bracket is firmly fixed.	Measuring ②: - Place the person to be measured underneath the Body Measuring Tape with the back on the wall. - Make sure the person's heels touch the wall and the head is in upright position. - Pull the measuring part down to the head of the person. - Reading mark is now indicating the length of the measured person. - Take the measuring part back to starting position. Cleaning: - Clean the plastic parts with a damp cloth if necessary. - Do not use any aggressive cleaning fluids. - Protect the metal band from moisture.
--	---

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ADE Germany GmbH