**ADE**

MZ10021

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## Operating instructions

### Scope of delivery:

- Body Measuring Tape
- Operating instructions

### Measuring:

- Pull the tape and wrap it around the body part.
- Insert the locking pin into the tape receptacle ①.
- Push and hold the Adjust button until the tape has fully retracted around the body part ②.
- Read the measurement at the marked point ③.

- Unlock pin.
- Fully retract tape. This will prepare the unit for the next measurement and ensures accuracy.

### Advice:

- It is recommended to take measurements in a relaxed state. Do not flex while taking the measurements.
- Measurements should be taken prior to any physical activity.
- Measurements should be taken on the right side of the body.

### Cleaning:

- Take care the circumference measuring tape is always cleaned to avoid infections.

- For cleaning use common disinfection products or soft cleaning fluids on a damp cloth.
- Do not use any aggressive cleaning material.

### Warranty

A two-year warranty from date of purchase will be done by repair or replacement once it is proved that the defects are caused by poor material or poor workmanship (please keep the receipt for presentation). All moveable parts such as batteries, cables, plug-in power supply units, rechargeable batteries etc. are excluded. Warranty does not apply to normal appearance of usage or sign of wear or any damages caused by accidents or misuse. Products which have been opened by unauthorized persons are automatically out of

warranty.

Customers abroad are requested to contact their local sales agent in the event of any warranty becoming applicable.

### Technical data

Measuring range: 50-1500 mm  
Graduation: 1 mm  
Dimensions: 80x55x20 mm  
Weight: approx. 45 g

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ADE Germany GmbH