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Operating Manual

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Bedieningshandleiding

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ADE

ADE Special

Teleskop- Längenmessstab

Telescopic Height Measuring Rod

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Telescopische lengtemeetstaaf

Estadiómetro telescópico

Estadiómetro telescópico

MZ10023-1

MZ10023-2

MZ10023-251211-Rev007-UM-de-e-fr-nl-es-pt

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English language version..... 7

Version française11

Nederlandse taalversie.....16

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Intended use and safety instructions

This symbol means: "Consult operating instructions". To achieve accurate measuring results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference and in reach for the user.

Your ADE measuring rod is a high-quality product and a length measuring instrument. To achieve the best possible results, please follow the instructions in this operating manual carefully.

The measuring rod is used to determine the height of adults and children in medical settings (e.g. doctors' surgeries, outpatient clinics, etc.).


It can be used either mounted on the wall or in combination with an ADE column scale. The measuring rod can be mounted on the front or back of the scale column. Together with the scale, you have a reliable and durable measuring and weighing station that helps you determine the health status of your patients.

Safety instructions

- Follow all instructions and notes in this operating manual.
- Before assembling and using the measuring rod, check it for damage. If damaged, do not use the device due to the risk of injury.
- Only attach the measuring rod to ADE column scales or to a wall using a measuring stick and spirit level.
- When the measuring rod is not being used for measuring, always swivel the measuring tongue down completely to prevent any risk of injury.

Carton content

- Telescopic measuring rod
- Mounting material for the two possible installation types (see image)
- User manual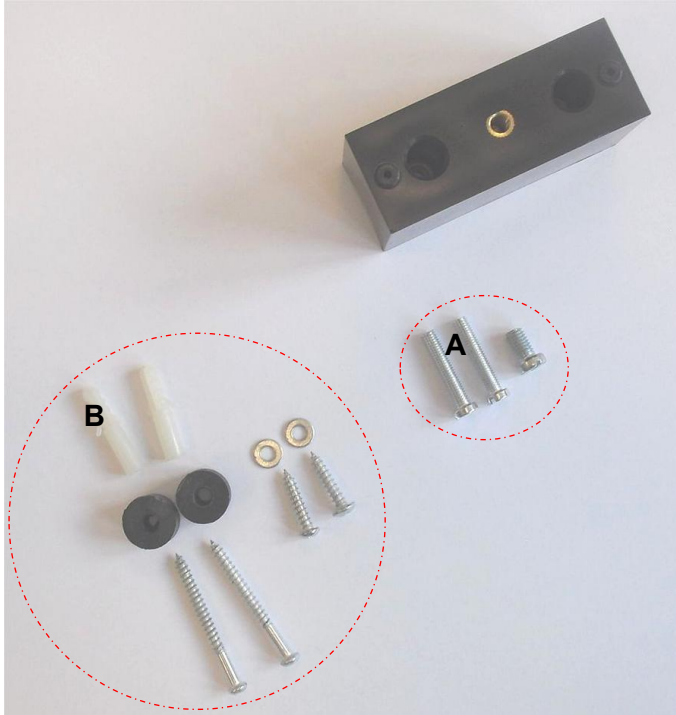

spacer block for mounting to the front or rear side of the column of a column type scale.

A: Fixation screws for fixing the measuring rod to the scale column.

B: Screw set for wall mounting incl. wall dowels, screws and washers.

Column scale assembly

- 1 spacer block for mounting to the front or rear side of the scale column
- 1 M6 screw
- 2 M5 screws

Wall assembly

- 2 spacer washers (black plastic)
- 2 screws, short (for mounting to wood or metal walls)
- 2 dowels S6
- 2 long screws (for mounting to concrete walls)
- 2 washers


Installation Instructions

Mounting the Measuring Rod to the Scale Column

Your ADE column scale is equipped with two threaded holes on the back of the column. These allow mounting of the spacer block for the measuring rod.

Position the spacer block over the threaded holes on the column and fasten it into the fixing bores **1** using the two M5 screws. Tighten both screws firmly.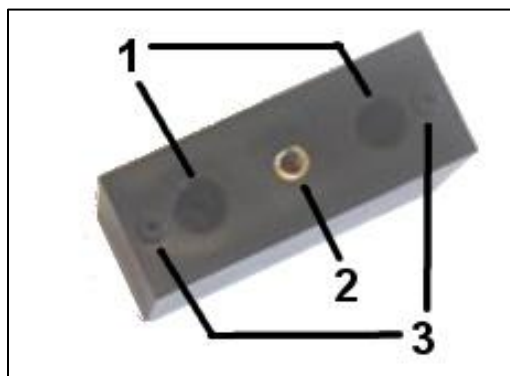

1: fixing bores

2: thread for fixing screw

3: locking pins

The spacer block additionally includes a threaded hole **2** for the measuring rod's mounting screw, as well as two positioning pins **3** spaced symmetrically. At the lower end of the measuring rod, on the printed side (measuring scale), there are corresponding holes for alignment. Position the measuring rod onto the spacer block so that the locking pins align with the corresponding holes. Secure the measuring rod using the M6 screw in the lower mounting hole. Tighten the screw firmly.

Mounting the Measuring Rod to the Wall


At the lower end of the measuring rod, on the printed side (scale), there are four holes. The lower three are intended only for column scale mounting and are not required here.

A through-hole is located approx. 1 cm below the beginning of the measuring scale (scale reading 60 cm). For correct measurement results, this hole must be positioned exactly 56 cm above the floor. (This offset compensates for the difference between the measuring tongue's contact surface and the reading mark on the scale.)

Use a measuring stick to mark exactly 56 cm above the floor on the wall. Extend the measuring rod fully.

Position the lower mounting hole of the rod (approx. 60 cm on the scale) over your 56 cm mark. Mark the upper mounting point of the rod by marking the wall through the upper hole (approx. 122 cm on the scale). Use a spirit level to ensure the rod is perfectly vertical. Drill the holes at the marked points and insert the two wall dowels.

Each mounting screw requires a washer. Slide a washer onto each screw and insert the screws through the holes (at approx. 60 cm and 122 cm). For final installation, place one black spacer washer on each screw from the back of the rod before tightening the screws into the wall plugs. Tighten both screws firmly.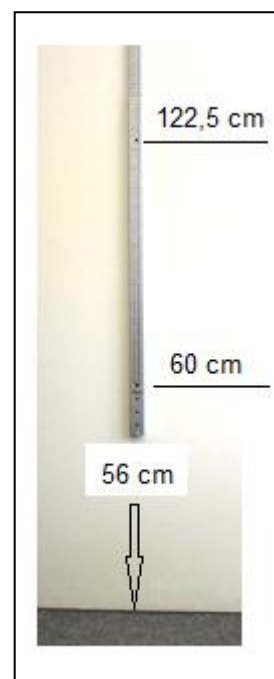

Operating the Measuring Rod


1. Move the measuring tongue to the horizontal position. It will audibly and noticeably lock into place.
2. Raise the measuring tongue above the person to be measured.
3. Have the person stand directly under the measuring tongue.

(illustration on principle; ill. showing different ADE product)

4. Lower the measuring tongue until it rests flat on the person's head.
5. Read the measurement at the red line located between the lower and upper part of the measuring rod.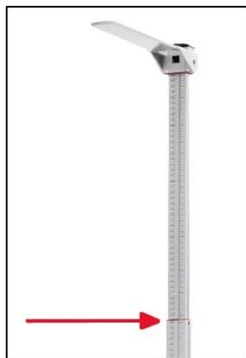

For measuring persons under 134 cm press the black button in the center of the measuring tongue and slide the tongue further down onto the lower section of the measuring rod. Read the measurement at the red line located at the lower edge of the measuring tongue.

6. When not in use, swivel the measuring tongue downward to prevent injury. A slight resistance is noticeable while swiveling the tongue.

Cleaning and Maintenance

For cleaning the measuring rod use a damp, soft and lint-free cloth. If necessary, use mild detergents only (e.g. household cleaner).

Do not use any aggressive, abrasive, acid or chlorine containing detergents or disinfectants.

If a surface disinfection becomes necessary, use an alcohol-based disinfectant and follow all dosing and handling instructions given by the respective manufacturer of the agent.

Warranty – Liability

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse, unsuitable or improper storage or inappropriate transport methods.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

Disposal of the equipment:

If the measuring rod has reached its end of service life, dispose of it properly and professionally in accordance with the country-specific laws and local regulations.

Technical Data

Measuring Range:	600 – 2100 mm
Graduation:	1 mm
Dimensions (WxHxD):	55 x 940 x 34 mm
Weight:	approx. 700 g

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ADE Germany GmbH