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## Installation and use of height rod

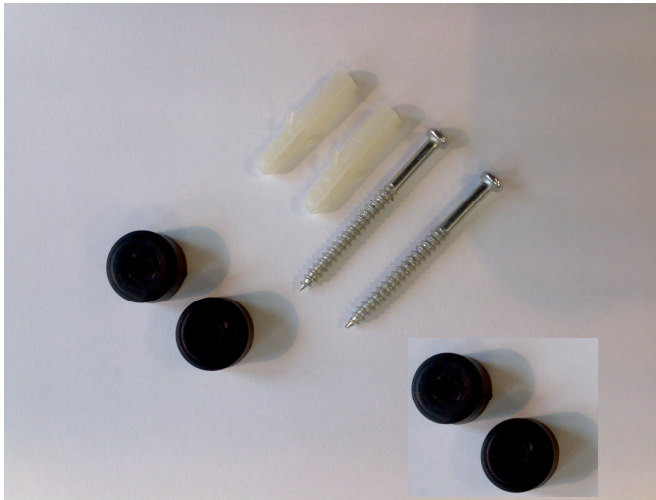
This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

Your ADE height measuring rod is a quality product. For measuring the length of adults and children the height measuring rod can either be used at your ADE column scale or can be fixed directly to the wall.

### **Carton content:**

- Telescopic measuring rod
- Foot positioner
- 2 spacers for the foot positioner
- 2 screws for connecting the measuring rod and the foot positioner
- Mounting material for fixing to the wall (see photo below)
- instruction manual

Screw set for wall mounting incl. wall dowels, screws and washers


### **Wall assembly:**

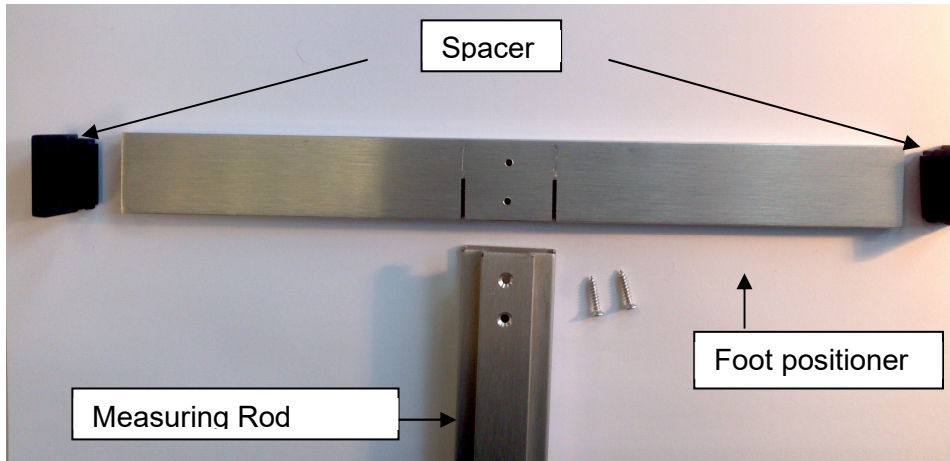- 4 spacers
- 2 screws (for assembling to wood or metal walls)
- 2 dowels S6

## Installation

### Assembling to the wall:

- 1) Place the black spacers into the open ends of the foot positioner. The spacer must be placed with the long distance piece on the side with the boreholes.
- 2) Push the foot positioner gently into the designated lower end of the measuring rod. Fix same with the two small screws.

- 3) Select a plane and level location at the wall.
- 4) Pull out the measuring rod until both boreholes are free.
- 5) Place the measuring rod with a water-level vertical at the wall.
- 6) Mark the drill holes at the wall through the wholes at the measuring rod.
- 7) Drill using a matching drill bit.
- 8) Insert the two anchors into the wall.
- 9) Insert now the two long screws through the holes into the measuring rod.
- 10) Put the round spacers over the screws between the rod and the wall.
- 11) Screw the screws into the anchors very tightly.
- 12) The foot positioner must be supported against the wall by the two black spacers.

### Safety advice

Follow the instructions of the Operating Manual.

After finishing the measurement make sure to fold the measuring tongue down to avoid danger of injury.

### Operating Instructions

- 1) Adjust the measuring tongue to a horizontal position.
- 2) Slide the measuring tongue up to over the length of the person to be measured.
- 3) Place the person to be measured under the measuring tongue.
- 4) Slide the measuring tongue down to the person's head.
- 5) Take the result from the readout point.

## **Warranty – Liability**

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

## **Technical Data**

Measuring Range:	600 – 2100 mm
Graduation:	1 mm
Dimensions (WxHxD):	55 x 940 x 34 mm
Weight:	approx. 700 g

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ADE Germany GmbH