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Electronic Flat scale | Báscula de suelo electrónica |  
Pèse-personne électronique plat | Bilancia da pavimento elettronica |  
Elektroniczna waga płaska | Elektronische Vlakke schaal

Deutsch

English

Español

Français

Italiano

Polski

Nederlands

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M316600

Operating instructions | Instrucciones de operación | Mode d'emploi  
Istruzioni per l'uso | Instrukcja obsługi

# Liebe Kundin, lieber Kunde !

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# Dear Customer!

You have chosen to purchase a high-quality ADE brand product, which combines intelligent functions with an exceptional design. This bathroom scale makes it possible for you to determine and monitor your body weight. The wealth of experience of the ADE brand ensures a high technical standard and proven quality.

Your ADE Team

## General information

---

### About this operating manual

This operating manual describes how to operate and maintain your product safely. Keep this operating manual in a safe place for future reference. If you pass this product on to someone else, pass on this operating manual as well.

Failure to observe this operating manual may result in injuries or damage to your product.

### Explanation of symbols

This symbol combined with the word DANGER warns of serious injuries.

This symbol combined with the word WARNING warns of moderate and mild injuries.

This symbol combined with the word NOTE warns of material damage.

This symbol indicates additional information and general notes.

# Table of Contents

---

General information .....	12
Intended use .....	14
Safety .....	14
Overview .....	16
Scope of delivery .....	16
Start up.....	17
Inserting/changing batteries .....	17
Choosing the unit of measurement .....	17
Setting up the scale .....	17
Weighing .....	18
Easy weighing .....	18
Mother-child function .....	18
Resetting the scale.....	18
Cleaning.....	19
Fault/Remedy.....	19
Technical data.....	20
Declaration of Conformity.....	20
Warranty .....	20
Disposal.....	21

## Intended use

---

This scale is used for weighing people. The maximum load capacity of 200 kg must not be exceeded.

## Safety

---

- This product may be used by children aged 8 and above, as well as by persons with reduced physical, sensory, or mental capacities or lacking experience and knowledge if they are supervised or have been instructed in the safe use of the product and understand the risks that may arise.
- Children must not play with the product.
- Cleaning and user maintenance must not be carried out by children, unless they are 8 years of age or older and are being supervised.
- Batteries, when swallowed, can be life-threatening. You should therefore keep the batteries and scale out of reach of small children. If a battery has been swallowed, medical assistance must be sought immediately.

### **DANGER for children**

- Keep packaging material away from children. There is a risk of choking if swallowed. Also pay attention that children do not pull the packaging material over their heads.

### **RISK of fire/burning and/or explosion**

- There is a risk of explosion if the batteries are replaced improperly. Replace the batteries only with the same or equivalent battery type. Pay attention to the "Technical data".
- Batteries must not be recharged, reactivated by other means, dismantled, thrown into fire, or short-circuited. Otherwise, there is an increased risk of explosion and leakage; furthermore, gases may escape!

- Avoid contact of the battery fluid with skin, eyes, and mucosa. In case of contact, immediately rinse the affected areas with plenty of clean water and consult a doctor.

## **WARNING of risk of injury**

- Do not step onto the outer edge of any one side of the scale. Risk of tipping!
- Do not step onto the scale with wet feet or when its surface is damp. Risk of slipping!
- If the scale falls down and shatters, there is a risk of injury from glass splinters.

## **BEWARE of material damage**

- Place the scale on a stable, even surface.
- Do not overload the scale. Pay attention to the "Technical data". Place yourself on the scale exclusively for weighing purposes.
- Protect the scale from strong shocks and vibrations.
- Remove the batteries from the scale when they are depleted or when you are no longer using the scale. You will thus avoid damage that may result from leakage of battery acid.
- When inserting and changing batteries, pay attention to the correct polarity (+/-).
- Do not subject the batteries to any extreme conditions, by – for example – storing them on radiators or under direct sunlight. Increased risk of leakage!
- Clean the battery and device contacts, as needed, before inserting the battery.
- Do not make any modifications to the product. Only allow repairs to be carried out by a specialist workshop. Repairs carried out improperly can considerably endanger the user.
- Do not place the scale in the vicinity of devices that may emit electromagnetic radiation, such as mobile phones, wireless devices, or the like. Doing so may lead to incorrect display indications or malfunctions.
- There are feet on the underside of the scale. Since floor surfaces are made from various materials and are treated with diverse care products, it cannot be completely excluded that some of these substances contain ingredients that may adversely affect and soften the feet of the device. If necessary, place a non-slip pad under the scale.

# Overview

---

- 1 UNIT button (underside):
  - Changing the unit of measurement
  - Activate the mother-child function
- 2 Display
- 3 Weighing surface
- 4 Battery compartment (underside)

## Scope of delivery

---

- Flat scale
- Battery type LR03 (AAA), 1.5 V  $\text{---}$  (4x)
- Operating Manual

# Start up

---

## Inserting/changing batteries

For use you need 4 batteries of type LR03 (AAA), 1.5 V (included). The battery compartment is located on the underside of the device.

1. Slide the battery compartment cover in the direction of the arrow and remove it.
2. Insert the batteries as indicated at the bottom of the battery compartment. Pay attention to the correct polarity (+/-).
3. Replace the battery compartment cover and push it against the direction of the arrow as far as it will go.

Suggestions for changing the battery:

- A battery change is necessary when “Lo” or nothing at all is shown on the display.
- Only use the type of battery that is specified in the “Technical data”.
- Always change all batteries at the same time.
- Dispose of old batteries in an environmentally appropriate way (see “Disposal”).
- Calibrate the scale, see “Setting up the scale”.

## Choosing the unit of measurement

The standard unit of measurement upon delivery is “kg” (kilogramme). However, the measurement result can also be displayed in the English unit “lb” (pound) or “st:lb” (stone:pound).

1. To switch on the scale, step on the middle of the weighing surface.
2. Use the UNIT button on the back of the scale to set the desired unit of measurement. Note the display.

## Setting up the scale

1. Place the scale on an even, firm and dry surface. Do not place the scale on a carpet.
2. First, briefly step onto the middle of the weighing surface to calibrate the scale. As soon as the display shows “0.0”, calibration is complete. This calibration must be performed after each battery change.

# Weighing

---

## Easy weighing

### **WARNING** against injuries

- Do not step onto the scale with wet feet or when its surface is damp. Risk of slipping!
- Position yourself on the scale. Stand still and do not hold on to anything. Shortly afterwards, the measured weight is displayed flashing. You can now step off the scale. The result is displayed for a few seconds (auto-hold function). After a few seconds the scales will deactivate automatically.

## Mother-child function

This function is intended for weighing infants and very young children.

Note the display: The symbols shown always indicate what to do next when weighing.

1. Switch on the scale by stepping on the weighing surface with a little force.
2. Press and hold the UNIT button until a person symbol is displayed at the bottom left.
3. Step on the weighing surface and weigh yourself. The result is displayed.
4. Now hold your child in your arms and weigh yourself together.  
After a short time, the child's weight is displayed, then yours again. After a few display cycles, in which both measurement results are displayed alternately, the scale switches off automatically.

To switch off the mother-child function:

1. Switch on the scale by stepping on the weighing surface with a little force.
2. Press and hold the UNIT button until the person symbol disappears from the display.

## Resetting the scale

---

Reset the scale to its default settings if unusual or obviously incorrect values are shown on the display.

1. Remove the batteries from the battery compartment and wait a moment.

2. Reinsert the batteries. Pay attention to the correct polarity of the batteries (+/-). After switching the scale on, you can weigh yourself as usual.

## Cleaning

### NOTE on material damage

- Do not immerse the scale in water or other liquids.
- Never use harsh, scouring or abrasive cleaning agents or the like to clean the scale. Otherwise, the surface could be scratched.

1. Wipe the scale as needed with a slightly damp soft cloth.
2. Disinfect the weighing surface from time to time using a suitable disinfecting agent (available at specialist retailers and pharmacies).

## Fault/Remedy

Fault	Possible cause
No function	<ul style="list-style-type: none"><li>▪ Are the batteries empty or inserted the wrong way around?</li></ul>
The scale appears to be displaying wrong weighing results	<ul style="list-style-type: none"><li>▪ Is the scale located on a firm, even surface?</li><li>▪ Is a device that can cause electromagnetic disturbances located in the vicinity of the scale?</li><li>▪ Remove the batteries from the scale and wait a moment. Reinsert the batteries.</li></ul>
"Err" is shown on the display	<ul style="list-style-type: none"><li>▪ The maximum capacity of the scale of 200 kg has been exceeded.</li></ul>
"Lo" is shown on the display	<ul style="list-style-type: none"><li>▪ The batteries are empty (see "Inserting/changing batteries").</li></ul>
"C" is shown on the display	<ul style="list-style-type: none"><li>▪ The weighing/measurement process was faulty: The scale is not stable or you are not standing still on the weighing surface during weighing. Repeat the process.</li></ul>

## Technical data

---

Model:	Electronic Flat scale M316600
Battery:	4x LR03 (AAA)/1.5 V $\text{---}$
Rated current:	$\leq 29$ mA
Measuring range:	max. 200 kg
Tolerance:	0 – 25 kg: 0.25 kg 25 – 100 kg: 0.5 kg 100 – 200 kg: 1.0 kg
Display scaling:	50 g
Scale dimensions:	approx. 338 x 340 x 36 mm
Weight:	approx. 1.6 kg
Distributor (no service address):	ADE Germany GmbH & Co. KG Neuer Höltigbaum 15 22143 Hamburg, Germany

Our products are constantly being developed and improved. For this reason, design and technical modifications are possible at any time.

## Declaration of Conformity

---

ADE Germany GmbH & Co. KG hereby declares that the M316600 electronic Flat scale complies with the 2014/30/EU and 2011/65/EU guidelines. This declaration will lose its validity if any modification is made to the scale without our consent.

Hamburg, March 2021  
ADE Germany GmbH & Co. KG

## Warranty

---

ADE Germany GmbH & Co. KG guarantees free remedy of defects due to material or manufacturing errors by repair or exchange for 2 years from the date of purchase.

In the event of a warranty claim, please return the product with the proof of purchase (specifying the reason for claim) to your dealer.

# Disposal

---

## Disposing of the packaging

Dispose of the packaging with similar materials. Dispose of cardboard with waste paper and of films with recyclable materials.

## Disposing of the product

Dispose of the product in accordance with the regulations that apply in your country.

### **Devices must not be disposed of in normal household waste.**

At the end of its life, the product must be disposed of in an appropriate way. In this way, valuable materials contained in the device will be recycled and the burden on the environment avoided. Bring the old device to a collection point for electronic waste or to a recycling depot. For more information, contact your local waste disposal firm or your local administrative body.

### **Batteries and rechargeable batteries should not be disposed of in the household waste.**

All batteries and rechargeable batteries must be brought to a local or district collection point or to the retailer. Such batteries and rechargeable batteries can thus be disposed of in an environmentally friendly way.

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