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Installation and use

This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

Your ADE height measuring rod is a quality product produced manufactured under the high standards and in accordance with national regulation in Germany.

Carton content:

- Telescopic measuring rod
- Steel platform (carton 360 x 460 x 75 mm) incl. 2 screws already mounted


Installation

Make the measuring rod and the platform out of the cartons and remove all packaging materials.

Unscrew the already pre-installed screws located at the bracket for the measuring rod.

Move the measuring rod gently inside the retaining bracket. Take care that the end of the rod is accurately being positioned over the aluminum block within.

Screw the two M6 screws via the boreholes through the steel bracket into the designated aluminum measurement rod.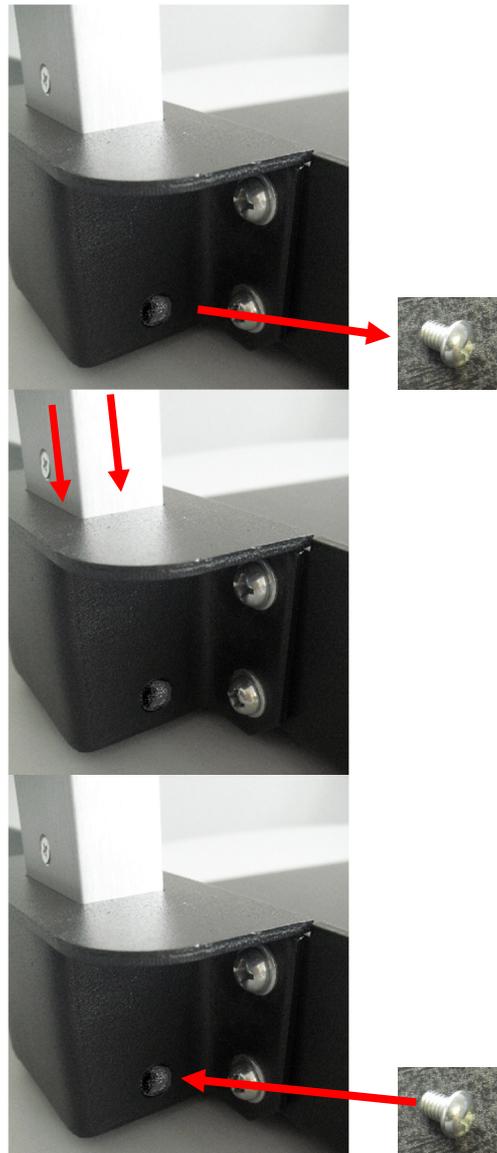

Safety

Follow the instructions of the Operating Manual in all respects. After measurement of patients make sure to fold the measuring tongue down to avoid danger of injury. Do not use any aggressive cleaning materials for cleaning; only use a damp cloth or conventional disinfectant.

Operation

- Place the patient on the footplate so that his or her back is to the column.
- Then pull out the length measuring unit at the head of the column as far as necessary. The length measuring rod is divided into three segments: two with size divisions and one without units (for the first 85 cm).
- Please make sure when pulling out that initially only the first segment is pulled out completely. Measurements of height from 85 cm to 140.5 cm can be taken with this.
- If the patient is taller than 140.5 cm, please also extend the second segment. The measuring range could be extended up to 210 cm.
- Unfold the measuring tongue and adjust the measuring rod so that the measuring tongue rests on the head of the person standing upright.
- In any case, the size reading takes place at the red mark at the height of 85 cm.

Warranty

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

Technical Details

Measuring range:	850 – 2100 mm
Graduation:	1 mm
Dimensions (WxHxD):	880 x 450 x 350 mm
Weight:	approx. 6,2kg

Manufacturer:

ADE Germany GmbH
Neuer Hoeltigbaum 15
22143 Hamburg / Germany
 +49 40 432 776-0
 +49 40 432 776-10
 info@ade-germany.de
 www.ade-germany.de

Händler / Retailer: