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This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

Your **ADE Electronic Baby Hanging Scale** has been manufactured for monitoring the weight of babies.

The weighing result is used as an essential indication of the weight development of neonates and babies. Make sure the baby is only weighed following strictly the instructions of this Operating Manual and is never left unattended.

In order to achieve the best possible results it is imperative to read and follow the Operating Manual in all respects. Always keep the Manual in a safe place next to the scale.

Scope of delivery

- Baby scale
- Operating instructions

Safety

- Handle your scale with care - it is a precision instrument.
- Never leave the infant / child unattended. Adult supervision is required at all times.
- Never leave the baby unattended hanging in the weighing trouser.
- This product is not a toy and should not be used as one. When not in use, store out of reach of children.
- Operate the unit only within the permissible ambient conditions.
- Do not operate the unit in hazardous or unstable environments.
- Locate the unit as far away as possible from equipment that generates electromagnetic or other interference.
- Portable and mobile RF communications equipment can affect electrical equipment. Before using high-frequency devices, observe their operating instructions.
- Use only approved accessories.
- Clean with a dry cloth, do not use harsh cleaning agents.
- Do not immerse the appliance in water or other liquids.

Keys

There are 4 function keys on this scale:

ON/OFF	To start or switch off the scale.
HOLD	To activate / deactivate the automatic hold function.
SET	To change the weighing mode from (kg / lb / oz) or to check the room temperature.
TARE	To reset the scale to zero.

Installation of scale

Insert batteries

- Please use two 2 x CR2032 (lithium) batteries.
- Open the battery compartment on the back side of the scale by loosening the fixing screws carefully.
- Insert the batteries according to the pole markings (+/-), with the + pole of both batteries pointing towards the bottom of the battery compartment.
- Please ensure that the battery compartment cover is in place and secured via the fixing screws before starting up the scale.
- Please observe the valid legal regulations in your country, when disposing used batteries.

In case scale will not be used for a long period of time take the batteries out to protect the scale and secure a long lifetime.

Operation

Weighing

To achieve a reliable result, make sure the scale is placed in a vertical position and no load is hanging at the hook.

1. Press the ON/OFF key to start the scale. During the automatic zero and function test you will see the indication `8888` in the display before `0.00` will come up in kg mode.
2. You may now use the scale. Make sure to use a weighing trouser or cloth supplied as optional extra by ADE.
3. To switch the scale off press the ON/OFF key again. Additionally, the scale is equipped with an automatic shut off function. Shutting down the scale after 60 seconds without use.

Tare function

Choosing this function, you may weigh the baby together with additional objects (e.g. nappy, vest etc.) and the weighing trouser or cloth without influencing the net weight result of the baby.

1. Press ON and wait for `0.00` indication on the display.
2. Hang the weighing cloth or trouser including other possible objects (nappy, vest etc.) at the hook.
3. Press TARE to reset the scale to zero.
4. Remove all weights from the hook. The scale will now display the weight as negative value and will hold the weight automatically.
5. Place the baby in the weighing trouser or cloth and hook it up to the scale. Press the HOLD key to restart the weighing function. The scale will show the next weight of the baby.
6. To cancel all values, press **TARE** again or switch off the scale.

Change unit

Switch the scale on and press the **SET** key short in order to change the units from kg into lb or oz. The scale will indicate the chosen units in the display.

Displaying the temperature

Switch the scale on and press the **SET** key and keep it pressed until the temperature is displayed. Press the **SET** key again to return into the weighing mode. The temperature is shown in °C only.

Hold function

The scale is equipment with an automatic hold function. The weight of the baby will automatically be locked in the display during each single weighing cycle. You can take the baby off the scale and read the weight on the display for 60 seconds. In order to unlock the weight, press the **HOLD** key again. The scale will fully automatically re-weight if the baby is still placed at the scale or will show zero.

Cleaning and maintenance

Cleaning

For cleaning just use a damp cloth or common disinfection products. Do not use any aggressive cleaning fluids. Make sure no water or cleaning fluid penetrates the scale.

Fault messages

If **LOBAT** is displayed, please replace the batteries. Always use two new batteries.

In case the scale does not show `0.00´ after having been started, press TARE to find the zero point.

In case of faulty results:

1. Make sure batteries are inserted correctly.
2. Make sure you are in the weighing mode you selected (kg / lb / oz).
3. Do not load the scale before the display shows `0.00´.

Disposal

Electronic devices are no domestic waste.

If the scale is out of use the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

Make sure only discharged rechargeable batteries respective rechargeable batteries with isolated poles will be disposed as otherwise short circuits may occur.

Warranty – Liability

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

CE Declaration of Conformity

Declaration of conformity

These scales have been manufactured in accordance with the harmonised European standards.
They conform to the regulations of the EC-directives listed below:

- EMC Directive 2014/30/EC
- RoHS Directive 2011/65/EC

in the currently valid versions.

This declaration loses its validity if modifications are made to the scale without our approval.

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 +49 40 432 776-0
 +49 40 432 776-10
 info@ade-germany.de
 www.ade-germany.de

Technical data

- Robust metal housing
- Capacity: 20 kg
- Graduation: 20 g
- Dimensions: 55 x 100 x 22 mm
- LC Display: 15 mm
- Weight approx.: 200 g

Dealer:

--

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ADE Germany GmbH